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TEL: 03-3817-8071 (直通) FAX: 03-3817-8075

2012.06

⚠ Attention Do not let ink from pens, newspapers, this manual or other sources come in contact with the manikin, as they cannot be cleaned the manikin skin.

Instruction Manual

M53B Prostate Examination Simulator

This simulator is designed to facilitate prostate examination training.
Do not use it for improper purpose. Please read the instruction carefully before use.

Features

- Ten different prostate models including normal prostate are prepared for training.
- 3 patient positions can be simulated: supine, prone and lateral position.
- Prostate models can be changed over smoothly by rotating the dial.
- Soft and life-like touch allows realistic trainings

Set Includes

Before your first use, ensure that you have all components listed below.

- ① Lumbar torso 1 piece
- ② Anus pad 1 piece
- Rotation unit: 10 prostate models are installed
 - Normal: 2 variations
 - Enlargement: 2 stages
 - Carcinoma: 5 variations
 - Prostatitis: 1 case
- ③ Vaseline 1 piece
- ④ Gloves 1 pair
- Storage case (no image)

● Consumable parts
11272-040 Anus pad

⚠ DOs and DON'Ts

DOs


- The materials for the models are special compositions of soft resin.
- Please handle them with the utmost care at all times.
- Please coat your finger(s) with vaseline, when you perform the training.
- Store the training set at room temperature, away from heat, moisture and direct sunlight.
- The rotation unit must be handled carefully when it is removed from the lumbar torso

DON'Ts

- Never wipe the simulator skin or pads with thinner or organic solvent.
- Don't mark on the phantom and models with pen or leave any printed materials in contact with their surface. Ink marks on the models won't be removable.

Training instructions

Select a case by rotating the dial.

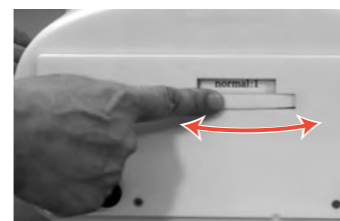
※ Set a case at the proper position that the dial stops with click sound.

○ Place the torso model as appropriate patient position.

Prone position

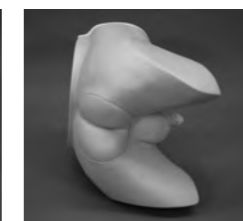Lateral position

Supine position

Replacement of the model parts

① Replacement of anus pad

1. Dismount: Gently, raise the edge around the unit and pull the unit straight off from the lumbar model.

2. Insert a new unit into the opening and fit the edge.

② Replacement of prostate models

1. Unscrew the rotation unit.


2. Withdraw the rotation unit carefully.

3. Slide off the prostate model which is to be replaced.

4. Slide in the new prostate model.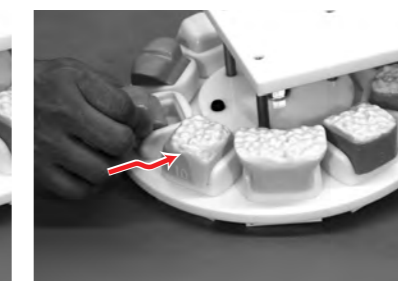

5. Restore the rotation unit into the body and make it upright. Rescrew and fix the unit.

KYOTO KAGAKU co.,LTD

URL • <http://www.kyotokagaku.com> e-mail • rw-kyoto@kyotokagaku.co.jp

■ Main Office and Factory

15 Kitanechoya-cho Fushimi-ku Kyoto 612-8388, Japan
Telephone : 81-75-605-2510
Facsimile : 81-75-605-2519

■ LA Office (for USA , CANADA and Mexico customers)

3109 Lomita Boulevard, Torrance, CA 90505, USA
Telephone : 1-310-325-8860
Facsimile : 1-310-325-8867

Please contact manufacturer for extra copies of this manual which may contain important updates and revisions.
Please contact manufacturer with any discrepancies in this manual or product feedback. Your cooperation is greatly appreciated.