
## Training instructions

Select a case by rotating the dial.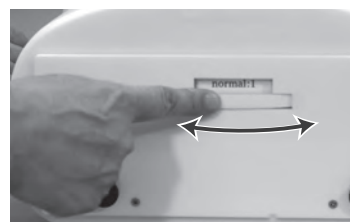

※ Set a case at the proper position that the dial stops with click sound.

○ Place the torso model as appropriate patient position.

Prone position


Lateral position

Supine position

## Replacement of the model parts

### ① Replacement of anal pad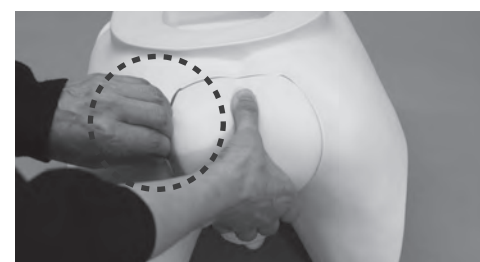

1. Dismount: Gently, raise the edge around the unit and pull the unit straight off from the lumbar model.

2. Insert a new unit into the opening and fit the edge.

### ② Replacement of prostate models



1. Unscrew the rotation unit.

2. Withdraw the rotation unit carefully.

3. Slide off the prostate model which is to be replaced. First, lift the outer edge of the prostate a little and then slide it off carefully. Do not force.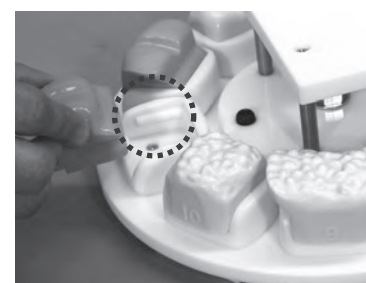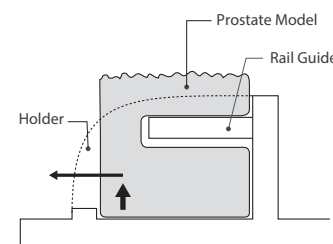

4. Slide in the new prostate model. Ensure to fit the groove to the rail on the holder. Then push in the outer edge of the model carefully into the frame.

5. Restore the rotation unit into the body and make it upright. Rescrew and fix the unit.

**KYOTO KAGAKU co.,LTD**

URL・<http://www.kyotokagaku.com> e-mail・[rw-kyoto@kyotokagaku.co.jp](mailto:rw-kyoto@kyotokagaku.co.jp)

#### ■ Main Office and Factory

15 Kitane-koya-cho Fushimi-ku Kyoto 612-8388, Japan  
Telephone : 81-75-605-2510  
Facsimile : 81-75-605-2519

#### ■ LA Office ( for USA , CANADA and Mexico customers )

3109 Lomita Boulevard, Torrance, CA 90505 ,USA  
Telephone : 1-310-325-8860  
Facsimile : 1-310-325-8867

Please contact manufacturer for extra copies of this manual which may contain important updates and revisions.  
Please contact manufacturer with any discrepancies in this manual or product feedback. Your cooperation is greatly appreciated.

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TEL: 03-3817-8071 (直通) FAX: 03-3817-8075

2017.12

Attention Do not let ink from pens, newspapers, this manual  
or other sources come in contact with the manikin,  
as they cannot be cleaned the manikin skin.

## Instruction Manual

# M53B Prostate Examination Simulator

This simulator is designed to facilitate prostate examination training.

Do not use it for improper purpose. Please read the instruction carefully before use.

### ● Features

- Ten different prostate models including normal prostate are prepared for training.
- 3 patient positions can be simulated: supine, prone and lateral position.
- Prostate models can be changed over smoothly by rotating the dial.
- Soft and life-like touch allows realistic trainings

## Set Includes

Before your first use, ensure that you have all components listed below.

① Lumbar torso 1 piece

② Anal pad 1 piece

Rotation unit: 10 prostate models are installed

Normal: 2 variations

Enlargement: 2 stages

Carcinoma: 5 variations

Prostatitis: 1 case

③ Vaseline 1 piece

④ Gloves 1 pair

Storage case (no image)

- Consumable parts  
11272-040 Anal pad

## ⚠ DOs and DON'Ts

### DOs

- The materials for the models are special compositions of soft resin.
- Please handle them with the utmost care at all times.
- Please coat your finger(s) with vaseline, when you perform the training.
- Store the training set at room temperature, away from heat, moisture and direct sunlight.
- The rotation unit must be handled carefully when it is removed from the lumbar torso

### DON'Ts

- Never wipe the simulator skin or pads with thinner or organic solvent.
- Don't mark on the phantom and models with pen or leave any printed materials in contact with their surface. Ink marks on the models won't be removable.