



In doing use of Suturing skin

1 Attachment and removal of a simulated skin

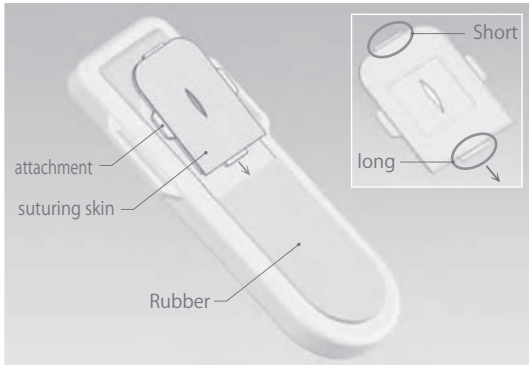

① To attach the skin unit, holding the side lugs of the plate, insert its longer tenon into the pit in the section of the larger rubber until it is all the way seated.

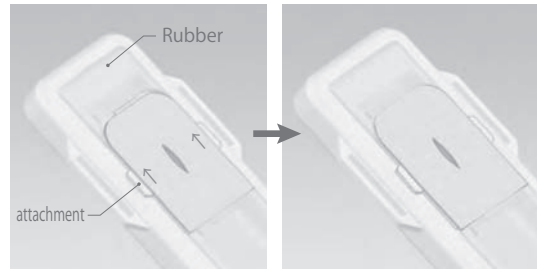

② Holding the lugs, slide the skin unit to the opposite direction so that its shorter tenon is inserted in the section of the smaller rubber.



③ When detach the skin unit, holding the lugs, slide the skin toward the larger rubber, and detach the tendons following the reverse procedure of its.



Caution

- Pay attention to direction, the upper and the lower of the Suture skin.
- The rubber can be damaged if the plate is forcibly inserted or removed without being slid.


2 Image Evaluation of a Suture Condition



○ Good




× Whole of the wound is not covered

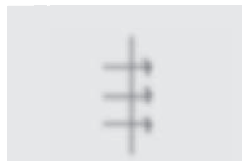

× Out of focus




× Overlapping tails
(Boxes will not be displayed.)

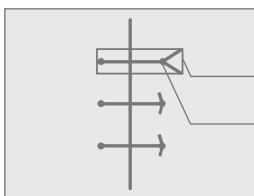

Needle insertion point evaluated by sensor camera
Actual insertion point

□ × Needle insertion point evaluated by sensor camera

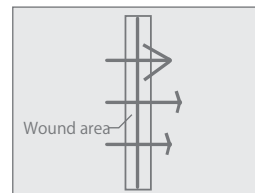

× A thread overlaps the wound area

KYOTO KAGAKU

e-mail ● rw-kyoto@kyotokagaku.co.jp
web ● <http://www.kyotokagaku.com>

● **Head Office and Factories:**
15 Kitanechoya-cho, Fushimi-ku, Kyoto, 612-8388, JAPAN
Tel: +81_75_605_2510 Fax: +81_75_605_2519

● **KyotoKagaku America Inc.** USA & Canada Inquiries & ordering
3109 Lomita Boulevard, Torrance, CA 90505-5108, USA
Tel: 1-310-325-8860 Fax: 1-310-325-8867