

**M59/63**

## Newborn Bathing and Nursery Care Model

M59 / M59B : Newborn Bathing and Nursery Care Model (Male)

M63 / M63B : Newborn Bathing and Nursery Care Model (Female)

M5963 / M5963B : Newborn Bathing and Nursery Care Model (Pair)

沐浴人形“柔” A形・B形


**⚠ Caution** Do not leave this manual in contact with manikin skin. Ink marks will be indelible, due to pigment inflation.

モデル表面に取扱説明書や印刷物などが直接触れないようにしてください。表面にインクが吸収され消えなくなります。

# Instruction Manual 取扱説明書

## Table of Contents もくじ

- English Manual..... P.1-P.3
  - Before you start
    - Manufacturer's Note / DOs and DONT's..... P.1
    - Set Includes..... P.2
  - Training
    - Training Skills..... P.2-P.3

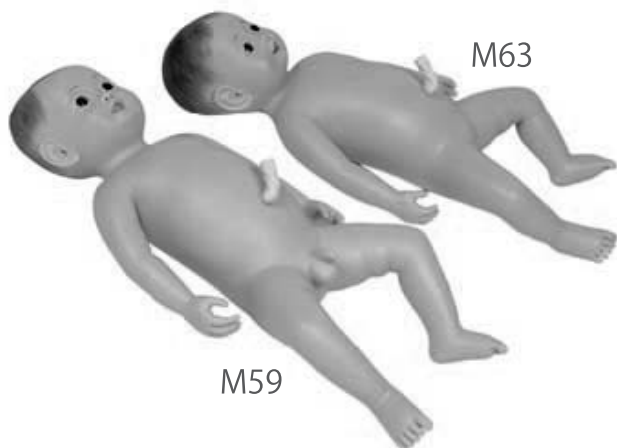

Movie Site



English Site

<https://youtu.be/DOsL5ApGDaE>

## ● Manufacturer's Note

This training model is a life-size seamless manikin of a 0-4-week old newborn infant. Taking a bath, holding the baby in your arms, feeding him/her, changing diapers, dressing him/her in clothes; all these functions become very real with this life-like model. The manikin has the same weight as a newborn baby, and the anterior and posterior fontanel also simulate those of a real human infant.

## ⚠ DOs and DON'Ts

### DOs

#### Handle with care

The model is made of special composition of soft resin. Please handle it with the utmost care like an actual baby at all times.

After each training session, dry the model with soft cloth and powder with talcum powder.

Store the model with its clothe on, covered by the supplied cloth, away from heat, moisture and direct sunlight.

#### Bottle Feeding and Diaper changing

Before feeding practice, put a diaper on the model. Feded fluid will be immediately discharged.

Perform the bottle feeding practice with water.

### DON'Ts

Never wipe the model by thinner or organic solvent.

Don't mark on the model with pen or leave any printed materials in contacted with their surface.

Ink marks on the models will not be removable.

## ■ Set Includes

Please ensure that you have all components listed all.



[M59 / M63]

- Newborn model
- Umbilical cord
- Cloth
- Feeding bottle
- Model Cover
- Instruction Manual

[M59B / M63B]

- Newborn model
- Umbilical cord
- Cloth
- Model Cover
- Instruction Manual

## ■ Training Skills

This training model is a life-size seamless manikin of a 0-4-week old newborn infant, which has the weight and balance of the newborn as well as palpable anterior and posterior fontanels. Both female and male manikins are available. Bathing, holding, feeding, changing diapers, dressing; every skill practice will be a true-to-life experience.

- Bathing
- Exersize
- Umbilical cord care
- Bottle-Feeding (M59/M63)
- Enema
- Diaper changing, clothing
- Physical measurement

### Bathing



The neck of the manikin is not rigid, replicating the status in newborn before learning to hold up his/her head. After training, Dry the model with soft cloth and put them on tulcam powder.

### Bottle-Feeding [M59 / M63]



The mouth and the urethral opening are interconnected and the model pass urine by feeding the milk.

\* M59B/M63B do not support bottle-feeding.

## Exercise



The limbs can bend and stretch; the supple material of the model allows trainees to practice baby care procedures just like real baby.

## Diaper changeing, clothing



The neck of the manikin is not rigid, replicating the status in newborn before learning to hold up his/her head. After training, Dry the model with soft cloth and put them on tulcam powder.

## Umbilical cord care



The umbilical cord is removable.



## Caution

Do not mark on the model and other components with pen nor leave printed materials contacted on its surface.  
Ink marks on the models will not be removable.



## 注意

モデル表面に印刷物などが直接触れないようにしてください。  
樹脂表面にインクが吸収されて消えなくなります。

- For inquiries and service, please contact your distributor or KYOTO KAGAKU CO., LTD.
- 本製品についてのお問い合わせや不具合等ございましたら、お手数ですがお買い上げの販売店、もしくは下記（株）京都科学まで御連絡ください。

## KYOTO KAGAKU co.,LTD

<http://www.kyotokagaku.com> e-mail: [rw-kyoto@kyotokagaku.co.jp](mailto:rw-kyoto@kyotokagaku.co.jp)

### ■ Main Office and Factory

15 Kitanechoya-cho Fushimi-ku Kyoto 612-8388, Japan  
Telephone : 81-75-605-2510  
Facsimile : 81-75-605-2519

### ■ 本社・工場

〒612-8388 京都市伏見区北寝小屋町15番地  
TEL: 075-605-2510 (直通)  
FAX: 075-605-2519

### ■ KyotoKagaku America Inc.

USA, Canada, and South America  
3109 Lomita Boulevard, Torrance, CA 90505, USA  
Telephone : 1-310-325-8860  
Facsimile : 1-310-325-8867

### ■ 東京支店

〒113-0033 東京都文京区本郷三丁目26番6号  
NREG本郷三丁目ビル2階  
TEL: 03-3817-8071 (直通)  
FAX: 03-3817-8075

The contents of the instruction manual are subject to change without prior notice.

No part of this instruction manual may be reproduced or transmitted in any form without permission from the manufacturer. Please contact manufacturer for extra copies of this manual which may contain important updates and revisions.

Please contact manufacturer with any discrepancies in this manual or product feedback. Your cooperation is greatly appreciated.

本書の内容は、予告なしに変更することがあります。本書の内容の一部もしくは全部を当社に無断で複写・転載することを禁じます。