

M72 Male Eldery Care Manikin

Feature

The joints are flexible to move.
The body is designed for waterproof.
It is good to practice for shampooing, brushing and drying hair.
It is good to practice for changing position, changing of cloth, cleaning, and moving in a wheelchair.
This is appropriate lightweight and carriageable by single person.
It is possible to change and to brush the denture. (for M72)
It is possible to hold the position naturally as like a elderly person.

Specification

	M72
Height	168cm
Weight	14kg
Hair	Wig
Material	Soft Resin within Metallic Framework

DOs

Handle with care
The model is made of special composition of soft resin.
Please handle it with the utmost care like an actual baby at all times.
After each training session, dry the model with soft cloth and powder with talcum powder.
Store the model with its clothe on, covered by the supplied cloth, away from heat, moisture and direct sunlight.

DON'Ts

Never wipe the model by thinner or organic solvent.
Don't mark on the model with pen or leave any printed materials in contacted with their surface.
Ink marks on the models will not be removable.

The pictures below are showing how they work with learners.



 **KYOTO KAGAKU co.,LTD**

<http://www.kyotokagaku.com> e-mail: rw-kyoto@kyotokagaku.co.jp

■ Main Office and Factory

15 Kitanechoya-cho Fushimi-ku Kyoto 612-8388, Japan
Telephone : 81-75-605-2510
Facsimile : 81-75-605-2519

■ KyotoKagaku America Inc.

USA,Canada,and South America
3109 Lomita Boulevard, Torrance, CA 90505 ,USA
Telephone : 1-310-325-8860
Facsimile : 1-310-325-8867