

# M83 Suture Practice Trainers -Full set-

## Instruction manual



### Contents

- Introduction  
Manufactures note / DOs and DON'Ts / Set includes . . . . . P.1
- Preparation  
Setting of skins . . . . . P.2

# Introduction

## Manufacturer's note

Suture Practice Trainers -Full set- is a trainer for teaching and learning basic suturing skills. 4 types of skins facilitate training in suturing of various body part.

### Attention

This Suture Practice Trainers -Full set- has been developed for the training of healthcare professionals only. Any other use, or any use not in accordance with the enclosed instructions, is strongly discouraged. The manufacturer cannot be held responsible for any accident or damage resulting from such use. Please handle the components carefully and refrain from subjecting to any unnecessary stress or wear.

### Features

- The artificial skin can be cut easily at any site to prepare a novel wound.
- The soft skins are made of durable materials.
- Simple and durable supporters keep the soft skins arched or maintain tension in the tubular models.
- Instruments for the practice session are included in the package.

## ⚠ DOs and DON'Ts

### DOs

- Handle the trainer and the components with care.
- Storage in a dark, cool space will help prevent the skin colours from fading.
- The skin may be cleaned with a wet cloth, if necessary, using mildly soapy water or diluted detergent.


### DON'Ts

- Do not let ink from pens, newspapers, this manual or other sources contact with the trainer, as they cannot be cleaned off the skin.
- Never use organic solvent like paint thinner to clean the skin, as this will damage the surface.
- Even if color on components' surface might be changed across the ages, this does not affect the quality of trainer's performance.
- Do not leave the tapes or dressing materials onto the components as the adhesive may be left and the surface becomes sticky.

## Set Includes

Before your first use, ensure that you have all components listed below.

- a. 1 supporter for abdomen
  - b. 1 supporter for blood vessels and intestinal tracts
  - c. 1 supporter for limbs
  - d. 2 abdominal skins
  - e. 2 limbs skins
  - f. 5 blood vessels
  - g. 5 intestinal tracts
  - h. 1 set of instruments  
(cable ties, needle-holders, suture needles, scissors, suture threads and tweezers)
- Instruction manual

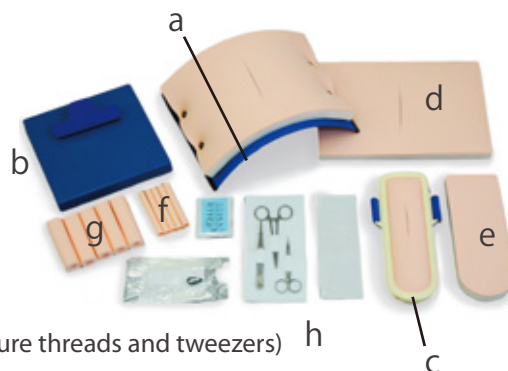

### 1 Abdominal skins

1. Place the skin on the supporter and screw the skin on the supporter at both side.



### 2 Limbs skins

1. Insert the skin to the support frame
2. Pull the skin to remove from the supporter frame.



### 3 Blood vessels and intestinal tracts

1. Fix the blood vessel or intestinal tract to the tube joint with the cable tie.



2. Pass the line of the tube joint through the hole of supporter then fix it with the cord stopper.



3. Clip another side of the blood vessel and intestinal tract.





## Caution

Do not mark on the model with a pen or leave printed materials in contact with the model surface. Ink marks on the model are not removable.

For inquiries and service, please contact your distributor or KYOTO KAGAKU CO., LTD.

Consumable  
parts

code	name
11219-030	1 abdominal skin
11219-050	5 blood vessels
11219-070	5 intestinal tracts
11219-010	5 limbs skins

## KYOTO KAGAKU co.,LTD

URL • <http://www.kyotokagaku.com> e-mail • [rw-kyoto@kyotokagaku.co.jp](mailto:rw-kyoto@kyotokagaku.co.jp)

### ■ Main Office and Factory

15 Kitanechoya-cho Fushimi-ku Kyoto 612-8388, Japan  
Tel : 81-75-605-2510  
Fax : 81-75-605-2519

### ■ Kyotokagaku America Inc. : USA, Canada and South America

3109 Lomita Boulevard, Torrance, CA 90505-5108, USA  
Tel: 1-310-325-8860  
Fax: 1-310-325-8867

The contents of the instruction manual are subject to change without prior notice.  
No part of this instruction manual may be reproduced or transmitted in any form without permission from the manufacturer.  
Please contact manufacturer for extra copies of this manual which may contain important updates and revisions.  
Please contact manufacturer with any discrepancies in this manual or product feedback. Your cooperation is greatly appreciated.