

Instruction Manual

M89 Oral Care Model

Effective oral care protects elderly people from pneumonia. The importance of basic oral hygiene and dentistry has been increasingly recognized in nursing and medical science.

● Features


1. This oral care model has a life-like sublingual and a replaceable denture.
2. This allows for practicing realistic oral cleaning procedure through use of simulated residue.
3. Settings to accommodate different patient postures necessary for practicing multiple procedures.
4. The replaceable denture is realistic, allowing practical training of placement and maintenance.

Set Includes

Before your first use, ensure that you have all components listed below.

a. oral model	1
b. base stand	1
c. denture	1
d. denture case	1
e. simulated residue	1
f. mirror	1
g. tooth brush	1
instruction manual	1

* The soft lower face and simulated residue are available as consumable/ replacement parts.

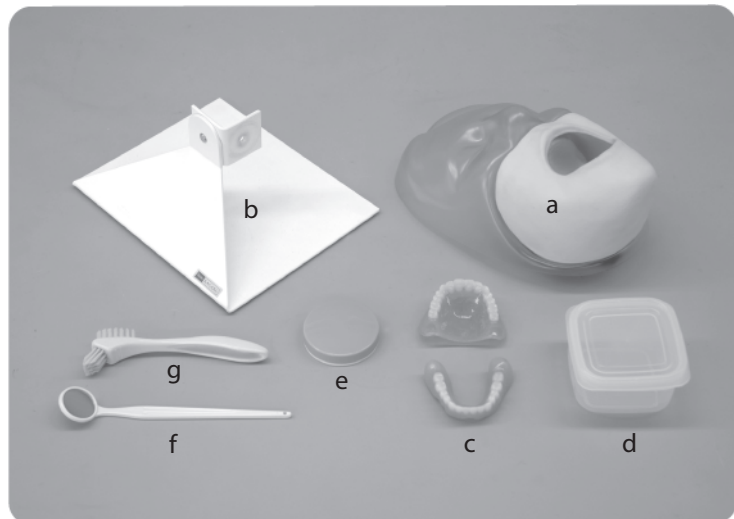

⚠ DOs and DON'Ts

DOs

- The materials for the manikin are special compositions of soft resin. Please handle them with the utmost care at all times.
- Dry the model after use, otherwise it might cause mold.
- Store the training set at room temperature, away from heat, moisture and direct sunlight.


DON'Ts

- Don't use the colored fluid, otherwise the manikin would be stained and it could not be cleaned off. Use the water.
- Never wipe the components with thinner or organic solvent.
- Don't mark on the components with pen or leave any printed materials in contact with their surface. Ink marks on the surface won't be removable.

Preparation/Setting



○ Assembly of the oral care model

1. Remove the black knob in the back of the soft lower face by turning it counterclockwise, and put the face manikin on the stand.
2. Then the face manikin by screwing the back knob clockwise.

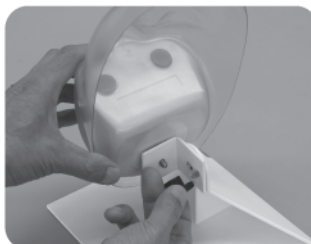

○ Adjusting of the position

1. Loosen the black knob counter clockwise to adjust the horizontal angle and then fix it by tightening the knob.
2. Sagittal angle can be adjusted.

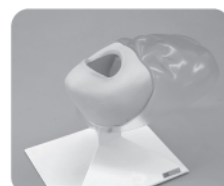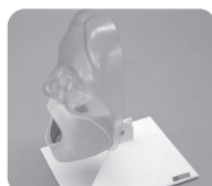

Caution

.....
Be careful not to turn the knob too tight.




Caution

.....
When the manikin face does not maintain its designated sagittal angle, tighten the screw on the stand by a driver.



Setting the Denture

1. Put the upper and lower dentures over respective gums so that dents at the back of dentures fit to the gums. Press the dentures to set in place.

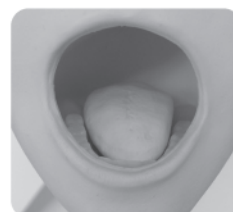

back side of the dentures

*Be careful not to tuck the tongue by the dentures.



KYOTO KAGAKU co.,LTD

URL • <http://www.kyotokagaku.com> e-mail • rw-kyoto@kyotokagaku.co.jp

■ Kyotokagaku Head Office and Factory

15 Kitanechoya-cho Fushimi-ku Kyoto 612-8388, Japan

Telephone : 81-75-605-2510

Facsimile : 81-75-605-2519

Please contact manufacturer with any discrepancies in this manual or product feedback. Your cooperation is greatly appreciated.

■ Kyotokagaku America Inc. : USA, Canada and South America

3109 Lomita Boulevard, Torrance, CA 90505, USA

Telephone : 1-310-325-8860

Facsimile : 1-310-325-8867