

### ⚠ WARNING

#### DOs

- ✓ Use each pad in **only a single session** (one practice).
- ✓ Avoid unnecessary extended exposure to air.
- ✓ Keep the pad away from high temperature, high humidity and direct sunlight.

#### DON'Ts

- ✓ Do not open the package until just before a practice.
- ✓ Do not keep the tubes (vein/ artery) filled with water.
- ✓ Do not leave any water on the pad.




1. Open the CVC pad.



1. Remove the lid behind the pad to become a landmark puncture pad.

**⚠ WARNING**  
Do not open the lid for "ultrasound use."

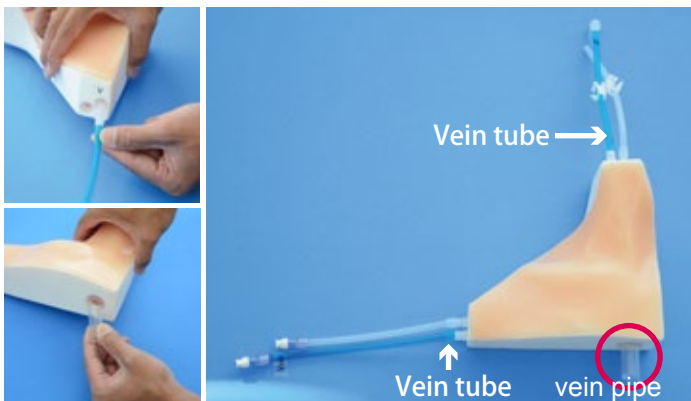

2. Connect the four tubes and a vein pipe to the CVC pad.

When attaching the vein pipe, please do not screw the pipe.



3. Connect the 50ml syringe to the tube and slowly inject with 50ml syringe twice (100ml of water) while raising the edge of the body to fill the tubes with liquid.



#### Temporary storage method

If you open the package by mistake;

This is a temporary measure only.

1. Apply the film tightly against the surface, to make sure there is no space or air trapped to cause evaporation of material moisture.
2. Put the pad in its case.
3. Store it in a sealed container or in an airtight bag that can be sealed.

2016/02/23

### ⚠ 注意

- パッドは、水溶性の材料を使用しており時間の経過とともに変形しますので、使用時以外は開封しないでください。
- 誤って開封した場合は、一時的な保管として密閉できる袋に入れてください。
- 高温多湿や直射日光を避けて保管してください。
- 穿刺部位を指で広げたり、強く押さないでください。
- 使用後は一般ごみとして廃棄してください。



1. パッド裏側のフィルムをはがし、パッドを取り出します。



1. ランドマーク穿刺の際は、裏側のキャップを外してください。

### ⚠ 注意

超音波ガイド下穿刺の場合は、キャップを外す必要はありません。



2. 血管チューブ4本(静脈：青色 / 動脈：透明 各2本)と、透明の筒(静脈パイプ)をパッドに取り付けます。


静脈パイプをとりつける際にはパイプをねじらないでください。



3. 水を入れた50mlのシリンジをチューブにつないで、ゆっくりと水を100ml注入し、血管チューブ内を模擬血液で満たします。

### 一時的な保管方法

原則1セッション(一回の実習)での使い切りを原則としておりますが、誤って内部の透明フィルムをはがしてしまった場合は、パッドを本体容器に入れ、フィルムを当てた上で密閉容器や密閉できる袋に保管してください。(但し一時的な保管となります)

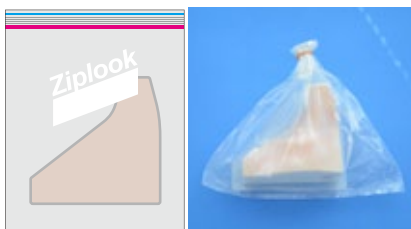

2020/05