M59/63

Newborn Bathing and Nursery Care Model

M59 / M59B : Newborn Bathing and Nursery Care Model (Male)

M63 / M63B : Newborn Bathing and Nursery Care Model (Female)

M5963 / M5963B : Newborn Bathing and Nursery Care Model (Pair)

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
⚠ Caution Do not leave this manual in contact with manikin skin. Ink marks will be indelible, due to pigment inflation.

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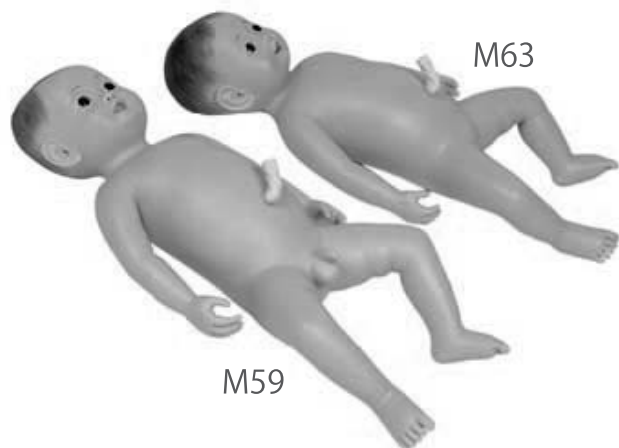● **Manufacturer’s Note**

This training model is a life-size seamless manikin of a 0-4-week old newborn infant. Taking a bath, holding the baby in your arms, feeding him/her, changing diapers, dressing him/her in clothes; all these functions become very real with this life-like model. The manikin has the same weight as a newborn baby, and the anterior and posterior fontanel also simulate those of a real human infant.

⚠ DOs and DON’ Ts	
<div><p>DOs</p><p><u>Handle with care</u></p><p>The model is made of special composition of soft resin. Please handle it with the utmost care like an actual baby at all times.</p><p>After each training session, dry the model with soft cloth and powder with talcum powder.</p><p>Store the model with its clothe on, covered by the supplied cloth, away from heat, moisture and direct sunlight.</p><p><u>Bottle Feeding and Diaper changing</u></p><p>Before feeding practice, put a diaper on the model. Fedeed fluid will be immediately discharged.</p><p>Perform the bottle feeding practice with water.</p></div>	<div><p>DON’ Ts</p><p>Never wipe the model by thinner or organic solvent.</p><p>Don’t mark on the model with pen or leave any printed materials in contacted with their surface.</p><p>Ink marks on the models will not be remov-able.</p></div>

■ Set Includes

Please ensure that you have all components listed all.

[M59 / M63]

- Newborn model
- Umbilical cord
- Cloth
- Feeding bottle
- Model Cover
- Instruction Manual

[M59B / M63B]

- Newborn model
- Umbilical cord
- Cloth
- Model Cover
- Instruction Manual

■ Training Skills

This training model is a life-size seamless manikin of a 0-4-week old newborn infant, which has the weight and balance of the newborn as well as palpable anterior and posterior fontanel. Both female and male manikins are available. Bathing, holding, feeding, changing diapers, dressing; every skill practice will be a true-to-life experience.

- Bathing
- Bottle-Feeding (M59/M63)
- Physical measurement
- Exercise
- Enema
- Umbilical cord care
- Diaper changing, clothing

Bathing

The neck of the manikin is not rigid, replicating the status in newborn before learning to hold up his/her head. After training, Dry the model with soft cloth and put them on talcum powder.

Bottle-Feeding [M59 / M63]

The mouth and the urethral opening are interconnected and the model pass urine by feeding the milk.

* M59B/M63B do not support bottle-feeding.

Exercise

The limbs can bend and stretch; the supple material of the model allows trainees to practice baby care procedures just like real baby.

Diaper changeing, clothing

The neck of the manikin is not rigid, replicating the status in newborn before learning to hold up his/her head. After training, Dry the model with soft cloth and put them on talcum powder.

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Caution

Do not mark on the model and other components with pen nor leave printed materials contacted on its surface.
Ink marks on the models will not be removable.

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The contents of the instruction manual are subject to change without prior notice.

No part of this instruction manual may be reproduced or transmitted in any form without permission from the manufacturer. Please contact manufacturer for extra copies of this manual which may contain important updates and revisions.

Please contact manufacturer with any discrepancies in this manual or product feedback. Your cooperation is greatly appreciated.

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- For inquiries and service, please contact your distributor or KYOTO KAGAKU CO., LTD.
KYOTO KAGAKU co.,LTD
■ KyotoKagaku America Inc.