

MW13/14/16 Difficult Airway Management Simulator Series

(Training Model, Demonstration Model, Bronchoscopy Model)

Parts change announcement

2019.06


As of May 2015, parts of the difficult airway management simulator series are changed as follows.

- Change on how to attach the replaceable tongue part


Previously, the anchor pin to connect the tip of tongue and the lower jaw was attached to the tongue side (11392-030). Now the design of the parts has changed so that the pin is fixed to the jaw side (11392-080). This improvement permits longer life of the tongue.

Previous model user needs the pin that is fixed to the jaw. Accordingly, one (1) pin is included in this parts set.

Existing part: 11392-030

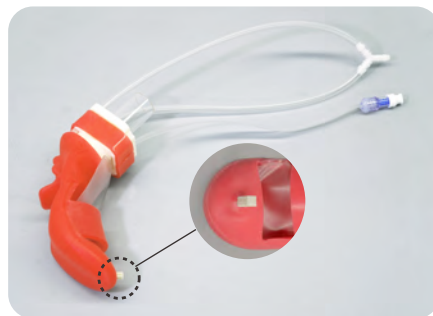

New part: 11392-080

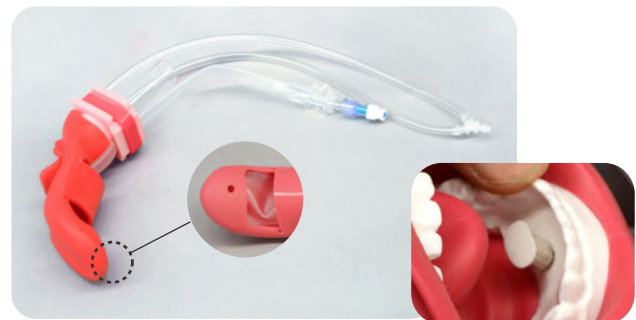


The pin attached to the lower jaw

The method to assemble and disassemble the new tongue part will differ from those found on the instruction manual. Please see the following instructions on how to attach and remove the tongue part from the lower jaw.

- **Instruction on tongue parts replacement (11392-080)**

Hook one side of the pin's head through the hole at the tip of the tongue. Then pull the tongue in the opposite direction to expand the hole. Once the hole is large enough, fit the pin through the hole to secure the tongue in place. To remove, pull the tongue to one side to expand the hole and remove from the pin.

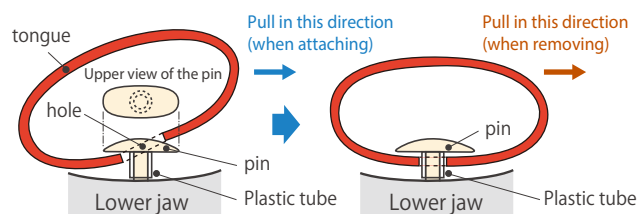

KYOTO KAGAKU co.,LTD

<http://www.kyotokagaku.com> e-mail: rw-kyoto@kyotokagaku.co.jp

■ **Main Office and Factory**

15 Kitanechoya-cho Fushimi-ku Kyoto 612-8388, Japan
Telephone : 81-75-605-2510
Facsimile : 81-75-605-2519

■ **KyotoKagaku America Inc.**

USA, Canada, and South America
3109 Lomita Boulevard, Torrance, CA 90505, USA
Telephone : 1-310-325-8860
Facsimile : 1-310-325-8867