


11251-350 Replacement of rectum valve

1. Screw the rectum part counter-clockwise.




screw the rectum part
counter-clockwise

2. Take out the valve and the ring, then change the new valve.
3. Before insertion of the valve into the rectum part, put the ring over the valve. And then screw the rectum on the connector of the genitalia unit.

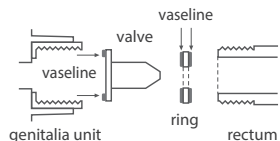

* Apply Vaseline to both sides of ring/ ring and also on the tip of the rectum container that touches the valve.

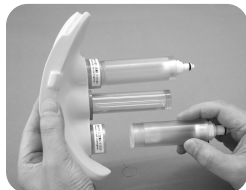

When the rectal valve is twisted, it may cause leakage of water. If it occurs, remove the rectal part, adjust the valve and set the part again.


11251-350 直腸弁の交換

1. 直腸部を左側に廻して取り外します。




左回しではずしてください

2. 中に入っている直腸弁とリングを取り出し、直腸弁を新しいものと交換します。
3. 取り付けは、直腸弁にリングを取り付け、直腸部に差し込んでから性器ユニットに取り付けます。

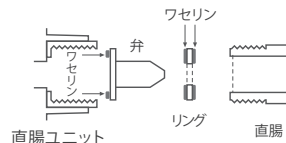

※ ワセリンをリングの両面及び直腸の弁と接する面に塗布します。

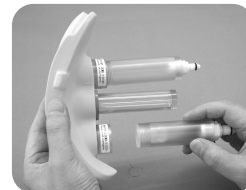

直腸弁がよじれて取り付けられていると浣腸液が漏れる場合があります。その場合は一旦取り外し、よじれないよう取り付けてください。

