



**Caution**

Do not mark on the model and other components with pen nor leave printed materials contacted on surface.

Ink marks on the models cannot be removed.

**MW37A**

# Episiotomy Suture Simulator

## Instruction Manual

### Contents

- Introduction ..... P. 1  
    Manufacturer's Note
- Before You Start ..... P. 2  
    Set Includes/ Precautions
- Training ..... P. 3
- After Training ..... P. 5





Episiotomy Suture Simulator has been developed for advancing the skills in Midwifery, and facilitates the training in Perineorrhaphy.

### ■ Features

- Suture training for episiotomy.
- Genital area excels in elasticity, durability and resilience to provide multiple training opportunity.

### ■ Training skills


- Episiotomy
  - Simple interrupted suture
  - Vertical mattress suture

### Read before use

This simulator has been developed for the training of medical and paramedical professionals only. Any other use, or any use not in accordance with the enclosed instructions, is strongly discouraged. The manufacturer cannot be held responsible for any accident or damage resulting from such use. Please use this model carefully and refrain from subjecting to any unnecessary stress or wear. Should you have any questions on this simulator, please feel free to contact our distributor in your area or KYOTO KAGAKU at any time. (Our contact address is on the back cover of this manual)

## Set Includes

Before you start, ensure that you have all components listed below.

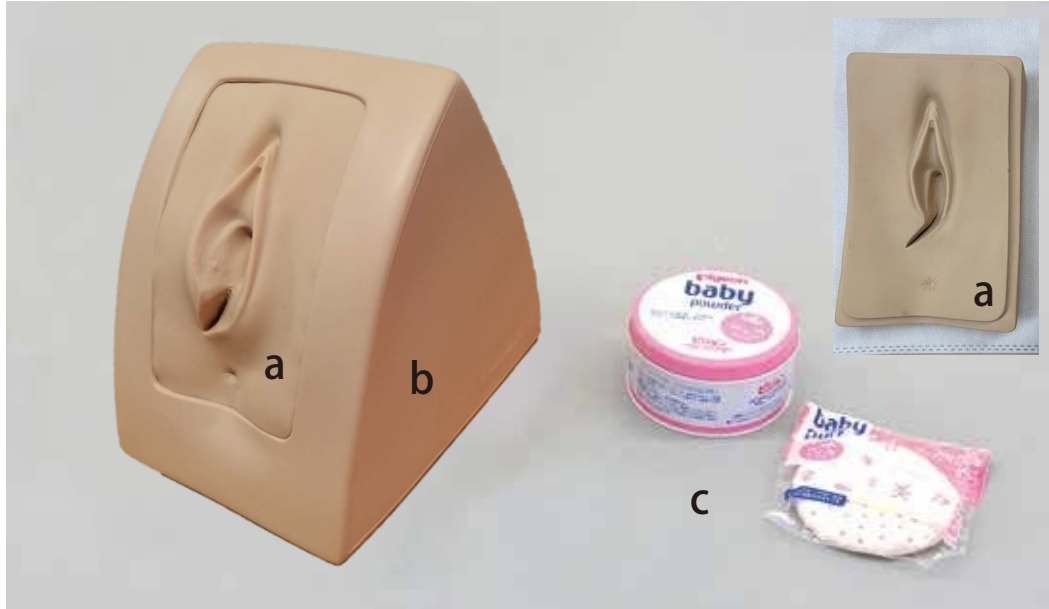

- |                      |   |                    |   |
|----------------------|---|--------------------|---|
| a. genitalia skin    | 2 | c. talcum powder   | 1 |
| b. supporting holder | 1 | instruction manual |   |

The type of talcum powder is subject to change without prior notice.

## ⚠ DO's and DON'Ts

### ● Handle the manikin and the components with care.

Be sure not to drop or add shock to the manikin and its components, since the product uses special resins and heavy, such shock may cause damage to the products or environment.

● Do not let ink from pens, newspapers, this manual or other sources contact with the manikin, as they cannot be cleaned off the manikin skin.

● Storage in a dark, cool and dry space will help prevent the skin colors from fading and product from deforming.

● Never use organic solvent like paint thinner or alcohol to clean the skin, as this will damage the simulator.

● The manikin skin may be cleaned with a wet cloth if necessary, using mildly soapy water or diluted detergent.

● Even if color on its surface might be changed across the ages, this does not affect the quality of its performance.

### PLEASE READ :

■ Always follow the handling and storage instruction below, to keep the material with its elasticity and durability.

#### AFTER USE:

Apply talcum powder on the genitalia skin, put on the supporting holder, cover them with included transparent bag and store them in the storage box.



○ **Do not let the parts contact with other resin items and printed materials.**

These soft material of the parts may be deformed by contact with other resin items and marked by contact with printed material and they cannot be repaired.

○ **Maintain proper temperature, humidity and avoid direct sunlight of the storage room of simulators.**

Be sure to monitor the indication sticker on the storage box of the product.

Store the simulator below 60 degrees C.

Change the storage place immediately when the temperature indicator shows color circles as below.

※The color circles are irreversible.

Before turned			After turned		
3F65	年	月貼付	3F65	年	月貼付
●	●	●	●	●	●
65	70	75	65	70	75



### Original Packaging

1. Take out the simulator from the storage box. The main unit is packed in the storage bag.



※Do not dispose this bag.  
It is necessary for storage.

The talcum powder and  
the puff are stored under  
the main unit.



### Training



Take out the main unit from  
the storage bag to practice  
Perineorrhaphy.

- Episiotomy
  - Simple interrupted suture
  - Vertical mattress suture

### Replacement of the Genitalia Skin

1. Put the finger into the gap between the genitalia skin and the holder to remove it, and attach the new skin. Put the genitalia skin into the pit of the supporting holder. Apply the talcum powder on the edge of skin to make it easy. Put the skin from directions to the holder. Ensure that the skin is sitting in the holder firmly.



### After Training

If the genitalia skin gets dirty after training, clean by wiping with soap or neutral detergent, and apply the talcum powder on it to store on the supporting holder putting in the storage bag definitely.



※Never use organic solvent like paint thinner or alcohol to clean the skin, as this will damage the skin and the supporting holder.

※The genitalia skin is made of the special material which is optimal for the suture training. If this skin is stored in contact with other resin products for a long time, the part might be deformed, or cause deterioration in the contacted resin products.



## Caution

Do not mark on the model and other components with pen  
nor leave printed materials contacted on surface.  
Ink marks on the models cannot be removed.

**For inquiries and service, please contact your distributor or KYOTO KAGAKU CO., LTD.**

The contents of the instruction manual are subject to change without prior notice.

No part of this instruction manual may be reproduced or transmitted in any form without permission from the manufacturer.

Please contact manufacturer for extra copies of this manual which may contain important updates and revisions.

Please contact manufacturer with any discrepancies in this manual or product feedback. Your cooperation is greatly appreciated.

---

### ■ Main Office and Factory (World Wide)

 **KYOTO KAGAKU co.,LTD**

**WEB**• [www.kyotogagaku.com](http://www.kyotogagaku.com)

**E-MAIL**• [rw-kyoto@kyotokagaku.co.jp](mailto:rw-kyoto@kyotokagaku.co.jp)

15 kitanechoya-cho Fushimi-ku Kyoto 612-8388, Japan

TEL: +81-75-605-2510

FAX: +81-75-605-2519