


Epidural Anesthesia Simulator



Read the full instruction manual carefully before use.

1 Connect the block to the drain pouch

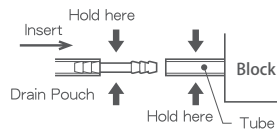


- 1) Ensure that the drain pouch is empty.
- 2) Hold the side tube of the puncture block and connect it to the connector of the drain pouch.

2 Connect the puncture block to the syringe

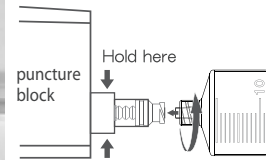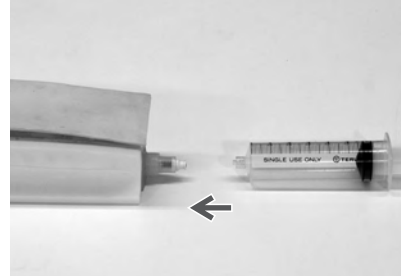


- 1) Fill the syringe with water.
- 2) Connect the tip of the syringe to the connector at the end of the block by screwing it clockwise.

3 Fill the Block with water

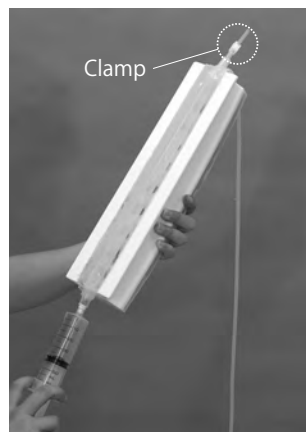

- 1) Open the clamp on the tube.
- 2) Hold the block upright with the open end of the tube upwards.
- 3) Push in the piston of the syringe and fill the block with water (approx. 25cc).
- 4) When the water surface reaches above the clamp, close the clamp firmly.
- 5) Unscrew the syringe.

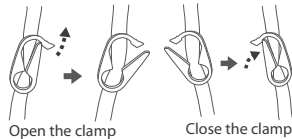

4 Set the block in the model




Put the block into the pit on the model so that the tube with clamp comes to the headside of the torso.

5 Set the body position



Set the supporting stand on the head side of the torso to keep it at (right or left) lateral position.



6 Ready

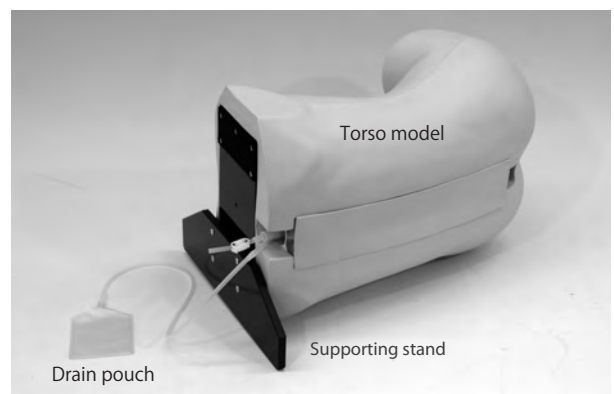

硬膜外麻酔シミュレータ



ご使用の前に必ず付属の説明書をお読みください。

1 穿刺パッドと排水バッグの接続

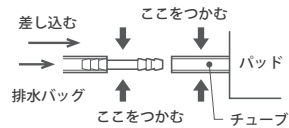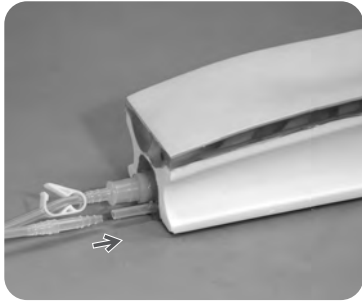

排水バッグ内に気泡や水滴が残っていないことを確認して、排水バッグの先端を穿刺パッドのチューブに接続します。

2 穿刺パッドとシリンジの接続

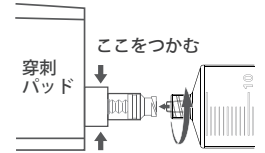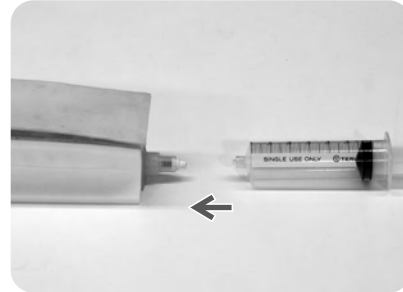


水を満たしたシリンジを穿刺パッドのコネクタを接続します。シリンジを時計回りに回すとロックされます。

3 パッド内に水を充填



パッドについているチューブのクレンメを開き、シリンジ側が下になるように穿刺パッドを持ってシリンジの水をクレンメ付近まで注入します。水を注入後、クレンメを閉じます。

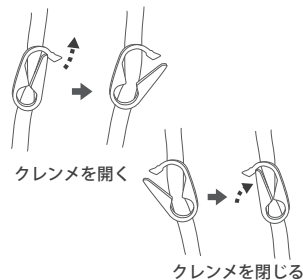

4 パッドをモデル本体にセッティング



穿刺パッドからシリンジをはずし、上下方向を注意してモデル本体にセットします。

5 体位の設定



シミュレータ本体の頭部側側面にマジックテープで固定ベースを貼り付け、手技に必要な体位を設定します。
(利き手が右利きの場合は左側臥位、右利きの場合は右側臥位に設定)

6 準備完了

これで準備完了です。

