

MW46



Caution


Do not mark on the model and other components with pen or leave any printed materials contacted on surface. Ink marks on the model cannot be removed.

Nail Specimen Collection Simulator

Instruction manual

Contents

- Introduction -----P.1
- Preparation
/ Training session-----P.2
- Consumable parts ----P.3

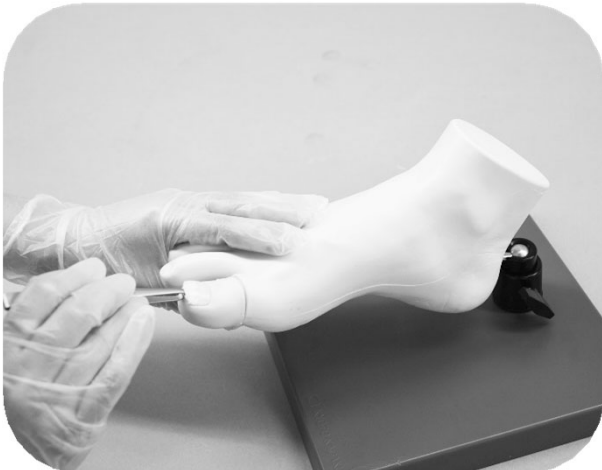

MW 46 is an educational tool that provides training for collection of nail specimen.

The product is designed for training of healthcare professionals. Usage for any other purpose or the way of use not described in this manual is strongly discouraged.

Training Skills

Collection of scales and crust

Features

Trimming fungal nails and specimen collection from appropriate sites Replaceable simulated nails for repeated practice

Read before use

The product is intended for use of medical and paramedical professionals only. Any other use, or any use not in accordance with the enclosed instructions, is strongly discouraged. The manufacturer cannot be held responsible for any accident or damage resulting from such use. Please use this product carefully and refrain from subjecting to any unnecessary stress or wear. Should you have any questions on this simulator, please feel free to contact our distributor in your area or KYOTO KAGAKU at any time. (Our contact address is on the back cover of this manual)


Introduction

Set includes

Before your first use, ensure that you have all components listed below.



- a. 1 foot model with swivel stand
- b. 20 simulated nails
- c. 10 simulated plantar hyperkeratosis unit

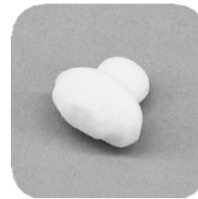

DOs and DON'Ts

- **Handle the manikin and the components with care**
Be sure not to drop or add shock to the manikin and its components, since the product uses special resins and heavy, such shock may cause damage to the products or environment.
- **Storage dark, cool and dry space will help prevent the product's colors from fading and product from deforming.**
- **The product may be cleaned with a wet cloth if necessary, using mildly soapy water or diluted detergent.**


- **Do not let ink from pens, oriented materials, this manual or other sources contact with the product's surface, as they cannot be cleaned off.**
- **Never use the organic solvent such as paint thinner to clean the skin, as this will damage the simulator.**
- **The color of the product's surface may change across the ages, this does not affect the quality of its performance.**

Preparation / Training Session


Preparation



Sites of planter hyperkeratosis

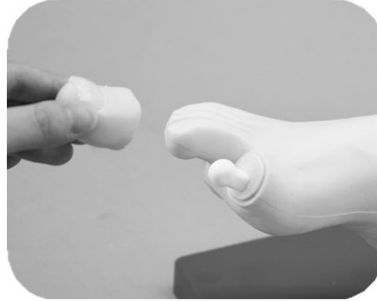

Attaching the nail

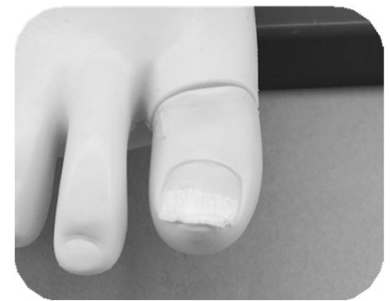

Installed nail

- 1) Attaching the training unit
Attach the planter hyperkeratosis and/or simulated nail to the foot model for training

The planter hyperkeratosis:
Insert the unit into one of the pits on the sole of the foot.

The simulated nail:
hold the toe and insert it firmly to the attaching site of the foot model

- 2) Adjusting the angle
Loosen the fixing screw while supporting the foot model. Adjust the angle to meet the training requirement. Tighten the screw to secure the model in place.



Fixing screw

Training Session

Training Skills

- Collection of scales and crust
- Trimming fungal nails

* Sample collection at callus using a metal depressor (not included in the set) is also possible.



Consumable parts

code	name
11427-010	simulated nail for MW46 (20 pcs)
11427-020	simulated plantar hyperkeratosis unit for MW46 (10 pcs)



11427-010
Simulated nail for MW46
(20 pcs)



11427-020 Simulated plantar
hyperkeratosis unit for MW46 (10 pcs)

For inquiries and service, please contact your distributor or KYOTO KAGAKU CO., LTD.

KYOTO KAGAKU co.,LTD

WEB:www.kyotokagaku.com e-mail:rw-kyoto@kyotokagaku.co.jp

Worldwide Inquiries & Ordering

Kyotokagaku Head Office and Factories:
15 Kitanechoya-cho, Fushimi-ku, Kyoto, 612-8388, JAPAN
Tel: +81-75-605-2510 Fax: +81-75-605-2519

All America regions:USA, Canada, and South America



WEB:www.kkamerica-inc.com E-MAIL:info@kkamerica-inc.com
3109 Lomita Boulevard, Torrance, CA 90505-5108, USA
Tel: 1-310-325-8860 Fax: 1-310-325-8867
(Toll free in North America:877-648-8195)

Europe, Russia & Africa

Kyoto Kagaku Europe GmbH.

TEL:+49-69-5060-28160
De-Saint-Exupery-Str.10, 60549 Frankfurt, Germany

2026/05

The contents of the instruction manual are subject to change without prior notice.

No part of this instruction manual may be reproduced or transmitted in any form without permission from the manufacturer. Please contact manufacturer for extra copies of this manual which may contain important updates and revisions.

Please contact manufacturer with any discrepancies, typos, or mistakes in this manual or product feedback. Your cooperation is greatly appreciated.