



Stool Specimen Collection Simulator



Instruction manual

Contents

Before Use	P.1
Preparation and Training	P.2

MW47 Stool Specimen Collection Simulator. This product is designed for training in safe stool specimen collection.

Read Before Use

MW47 has been developed exclusively for the training of medical and allied healthcare professionals. Any use other than its intended educational purpose, or any use not in accordance with this manual, is not recommended. KYOTO KAGAKU assumes no responsibility for any accident, injury, or damage resulting from improper use of this product. Handle the product with care. Do not subject it to unnecessary force, impact, or excessive wear. If there are any questions regarding this product, please contact the local distributor or KYOTO KAGAKU. Contact information is provided on the back cover of this manual.

Features

- Training for correct rectal swab insertion, including proper angle and depth
- Scenario-based training with consideration for patient dignity and modesty

Introduction

Set Includes

Verify that all components are included before us



- 1 Lumbar torso
- 1 Rectum unit
(pre-installed)
- 1 Lubricant (petroleum
jelly)

Precautions

- **Handle the simulator with care**
Dropping the simulator or subjecting it to strong impacts may cause damage.

- **Do not place printed materials on the product surface.**
Do not write on the product with a ballpoint pen, permanent marker, or other writing instrument. Ink may be absorbed into the resin surface and become impossible to remove.

- **Do not disassemble the product.**
Disassembly may result in damage.

- Store away from high temperature, high humidity, and direct sunlight. Improper storage may cause deterioration or damage.

- **Discoloration**
Discoloration may occur during long-term storage or as a result of natural aging. This does not affect product performance.

- **Cleaning**
Wipe the product with water or a mild detergent. If necessary, wipe with alcohol. Never use organic solvents such as thinner.

Preparation / Training Session




Replacing the Rectum Unit

The rectum unit is pre-installed in the lumbar torso. Replace the rectum unit if the anus or rectum becomes damaged.

Insert fingers around the edge of the rectum unit and remove it from the lumbar torso.


Align the rectum unit with the recess in the torso, then press it into place while guiding the surrounding edge by hand until it fits securely.

Apply a thin layer of the supplied lubricant (petroleum jelly) from the anal opening to the inside of the rectum before insertion to facilitate smooth insertion of the swab.

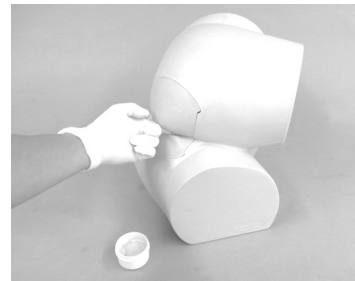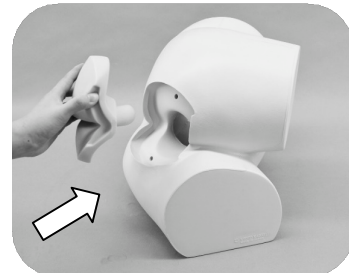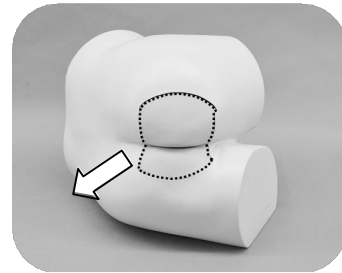

Training


Stool specimen collection from the rectum

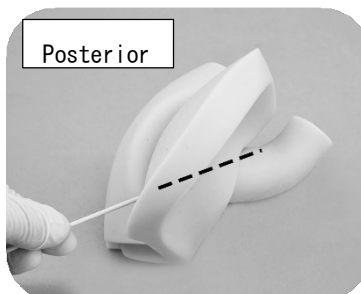

○

Insertion Guidelines

Insert the swab toward the patient's back. If resistance is encountered, the swab may be contacting the anterior wall of the rectum.

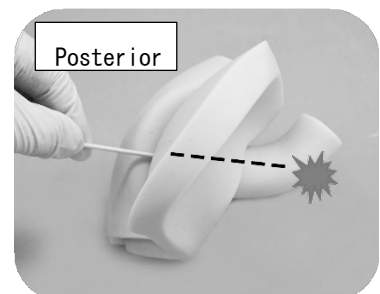

×

Consumables

Part No.	
11428-010	Rectum unit x 1



11428-010 Rectum unit



WEB: www.kyotokagaku.com e-mail: rw-kyoto@kyotokagaku.co.jp

Worldwide Inquiries & Ordering

Kyotokagaku Head Office and Factories:

15 Kitanecho-cho, Fushimi-ku, Kyoto, 612-8388, JAPAN

Tel: +81-75-605-2510 Fax: +81-75-605-2519

All America regions: USA, Canada, and South America



WEB: www.kkamerica-inc.com E-MAIL: info@kkamerica-inc.com

3109 Lomita Boulevard, Torrance, CA 90505-5108, USA

Tel: 1-310-325-8860 Fax: 1-310-325-8867

(Toll free in North America: 877-648-8195)

rw-kyoto@kyotokagaku.co.jp

Europe, Russia & Africa

Kyoto Kagaku Europe GmbH.

TEL: +49-69-5060-28160

De-Saint-Exupery-Str. 10, 60549 Frankfurt, Germany

The contents of the instruction manual are subject to change without prior notice.

No part of this instruction manual may be reproduced or transmitted in any form without permission from the manufacturer. Please contact manufacturer for extra copies of this manual which may contain important updates and revisions.

Please contact manufacturer with any discrepancies, typos, or mistakes in this manual or product feedback. Your cooperation is greatly appreciated.

2026/07