

MW67



Caution

Do not mark on the model and other components with pen or leave any printed materials contacted on surface. Ink marks on the model cannot be removed.


11460-100

Anorectal Manometry Examination Trainer

Table-top Training Unit

Instruction manual

Contents

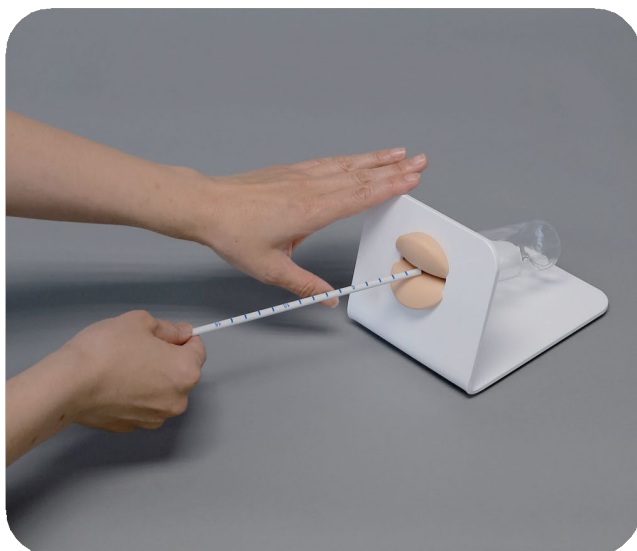


Set Includes.....	P.1
Handling Precautions.....	P.1
Preparation.....	P.2
Training Session.....	P.2
Replacement Part.....	P.3

MW67 11460-100 has been developed exclusively for the training of medical and allied healthcare professionals. Any use other than its intended educational purpose, or any use not in accordance with this manual, is not recommended. KYOTO KAGAKU assumes no responsibility for any accident, injury, or damage resulting from improper use of this product. Handle the product with care. Do not subject it to excessive force, impact, or unnecessary wear. If there are any questions regarding this product, please contact the local distributor or KYOTO KAGAKU. Contact information is provided on the back cover of this manual.

Set includes

Set Includes

Verify that all components are present before use.

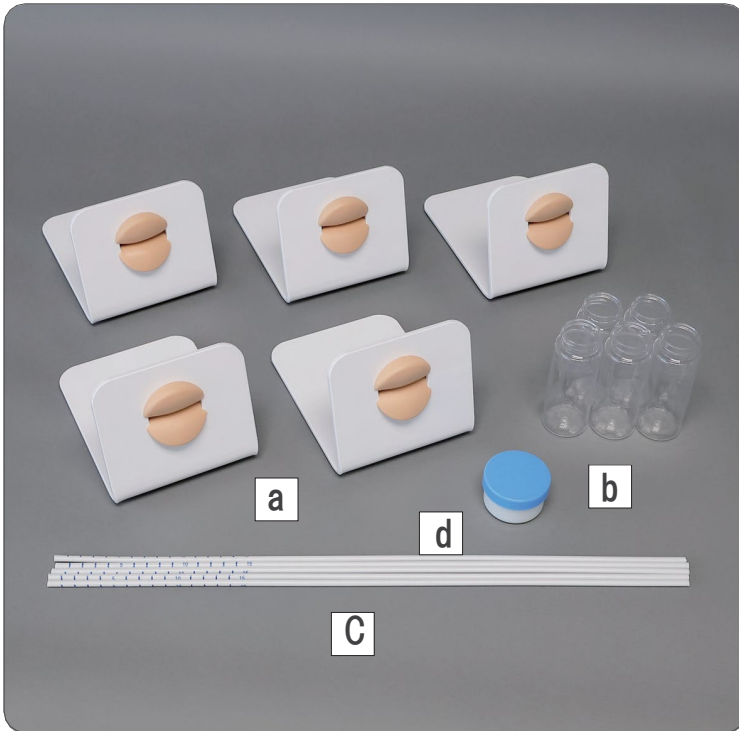

- a. 5 Table-top training unit
- b. 5 Simulated rectum
- c. 5 Simulated pressure transducer
- d. 1 Petroleum jelly (Vaseline)

Handling Precautions

- | | |
|---|--|
| <ul style="list-style-type: none">• Handle the simulator with care
Dropping the simulator or subjecting it to strong impacts may cause damage. | <ul style="list-style-type: none">• Do not place printed materials on the product surface.
Do not write on the product with a ballpoint pen, permanent marker, or other writing instrument. Ink may be absorbed into the resin surface and become impossible to remove. |
| <ul style="list-style-type: none">• Do not write on the product with ballpoint pens or permanent markers. Ink may be absorbed into the resin surface and become impossible to remove. | <ul style="list-style-type: none">• Store away from high temperature, high humidity, and direct sunlight.
Improper storage may cause deterioration or damage. |
| <ul style="list-style-type: none">• Discoloration
Discoloration may occur during long-term storage or as a result of natural aging. This does not affect product performance. | <ul style="list-style-type: none">• Cleaning
Wipe the product with water or a mild detergent. If necessary, wipe with alcohol. Never use organic solvents such as thinner. |
| <ul style="list-style-type: none">• Store the product only after it has dried completely. Store in a well-ventilated location after use. Storing the product while wet may result in mold growth. | |

Preparation / Training Session

Preparation (Installing the Simulated Rectum)

Screw the simulated rectum (bottle) onto the threaded fitting on the rear of the anal unit.


*Notes: The anal unit (soft resin) is permanently attached to the base. Do not attempt to remove it.



Training Session

Pressure transducer insertion


*Apply petroleum jelly (Vaseline) to the anal unit before inserting the simulated pressure transducer.

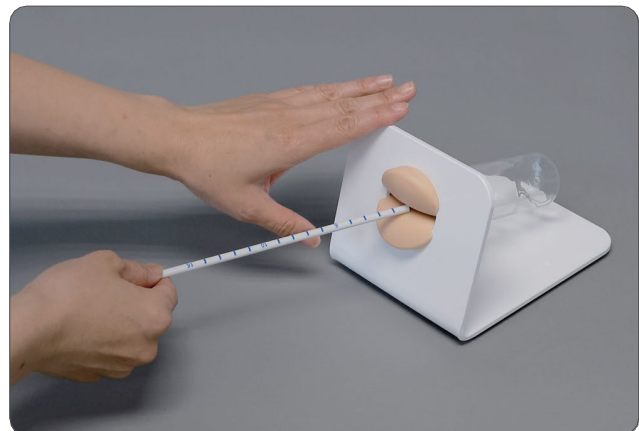

Scale markings on the simulated pressure transducer allow verification of the insertion depth.


The transparent simulated rectum also allows visual confirmation of the insertion depth from the rear of the product.

*After training, gently wipe off any remaining petroleum jelly with a well-wrung soft cloth. *Do not use alcohol or other organic solvents.

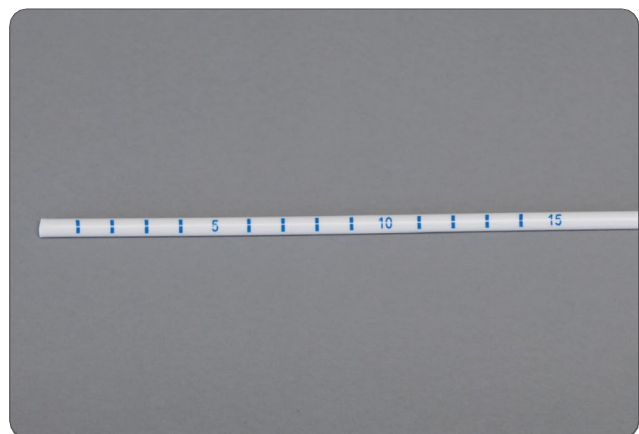

Replacement Part

Part No.	
11460-010	Simulated Pressure Transducer

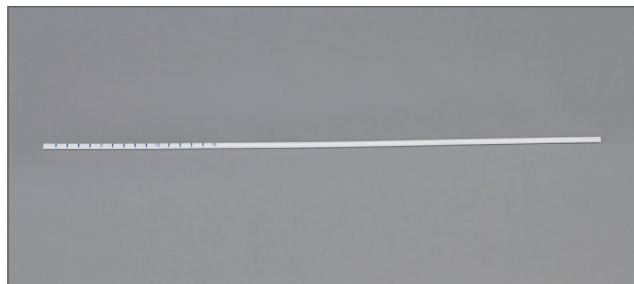

11460-010 Simulated Pressure Transducer



WEB: www.kyotokagaku.com e-mail: rw-kyoto@kyotokagaku.co.jp

Worldwide Inquiries & Ordering

Kyotokagaku Head Office and Factories:

15 Kitanecho-cho, Fushimi-ku, Kyoto, 612-8388, JAPAN

Tel: +81-75-605-2510 Fax: +81-75-605-2519

All America regions: USA, Canada, and South America



WEB: www.kkamerica-inc.com E-MAIL: info@kkamerica-inc.com

3109 Lomita Boulevard, Torrance, CA 90505-5108, USA

Tel: 1-310-325-8860 Fax: 1-310-325-8867

(Toll free in North America: 877-648-8195)

Europe, Russia & Africa

Kyoto Kagaku Europe GmbH.

TEL: +49-69-5060-28160

De-Saint-Exupery-Str. 10, 60549 Frankfurt, Germany

The contents of the instruction manual are subject to change without prior notice.

No part of this instruction manual may be reproduced or transmitted in any form without permission from the manufacturer. Please contact manufacturer for extra copies of this manual which may contain important updates and revisions.

Please contact manufacturer with any discrepancies, typos, or mistakes in this manual or product feedback. Your cooperation is greatly appreciated.

July.2027