




Don't mark on the model with a pen or leave printed materials on its surface.
Ink marks on the model will be irremovable.

MW72



Instruction manual

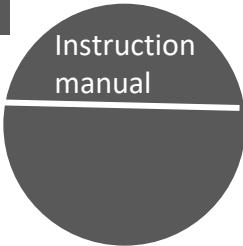

Strap-on Ostomy Care Trainer

Caution

<ul style="list-style-type: none"> ● Don't mark on the model with a pen or leave printed materials on its surface Any ink marks will be irremovable. 	<ul style="list-style-type: none"> ● Clean using neutral detergent or alcohol Use water, neutral detergent, or, for stubborn stains, alcohol to clean. ※ Do not use organic solvents
<ul style="list-style-type: none"> ● Keep away from heat, moisture and direct sunlight Wrong storage may lead to discoloration. 	<ul style="list-style-type: none"> ● The model might change color After time, the model may change color.
<ul style="list-style-type: none"> ● Let the model dry after training Dry using a soft towel after cleaning. When it's damp, let it dry in the shade. Inadequate drying may lead to molding. 	

■ Consumables

11465-040 simulated feces (set of 20)


■ Replacement parts

11465-010 replacement skin

11465-050 replacement strap

Set includes

Please check in advance if the set is complete.

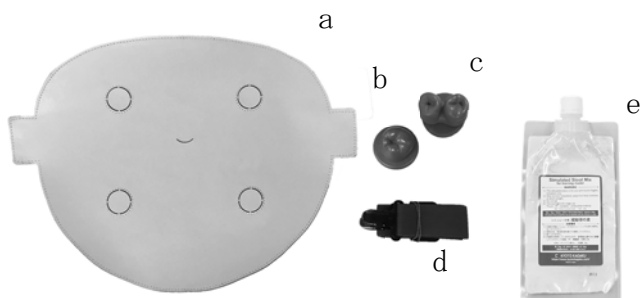

- a. skin 1
- b. end stoma 1
- c. loop stoma 1
- d. strap 1
- e. simulated feces 1

※ in case of the set of 5 simulators, 5 pieces each are included




Please visit our website for more details.
www.kyotokagaku.com



Preparation

1. Cut a hole


Cut a hole in the skin with scissors where you want to place the stoma.

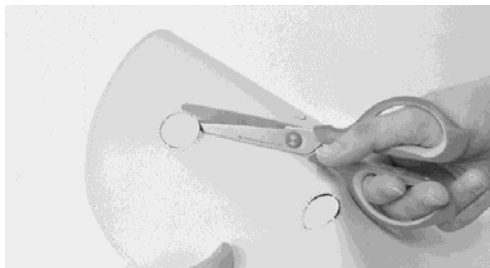

2. Attach a stoma

Squeeze the stoma from the side to easily attach it to the hole from the front side. Do not try to attach or detach the stoma from the back side.

When detaching, softly press from the back side if needed.

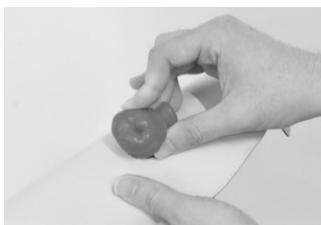

3. Attach the strap

The clip on the strap opens when bent and closes when returned to its original position. Align the navel of the model with the navel of the wearer and adjust the straps accordingly. When training stoma cleaning, we recommend wearing a water-proof apron or sheet beneath the model to avoid stains.

bend



return position




Preparing the simulated feces

1. Add water to the simulated feces (roughly 80ml)

By using more or less water, the viscosity can be adjusted.

※If there is too little water it will become hard to mix

※there is a chance that the simulated feces powder is damp before opening but this has no effect on the usage

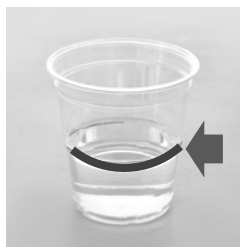

2. Shake

Close the bag while leaving some air inside and shake it thoroughly in all directions (minimum 30 seconds).

※it will not mix well if you try to mix with your hands



3. Pour the mixture into the stoma pouch

Fill the simulated feces into the stoma pouch.

Tip

Do not try to mix with your hands.

Stains on clothes

Wash off any simulated feces on your clothes immediately with neutral detergent.

※the materials used for the simulated feces are similar to those used in make up products and are not harmful to the human body

Video instruction



YouTube

<https://youtu.be/Fjti7kcQUng>