Caution

Don't mark on the phantom with pen or leave printed materials on its surface.  
Ink marks on the phantom will be irremovable.

MW74


11479-000

# Cesarean Section Unit

Instruction  
Manual

## Contents

General Information.....	1
Product Features and Precautions .....	1
Set Includes .....	2
Specifications .....	2
Training .....	3-4
Preparation.....	3-4
After Training.....	5
Disassemble and Storage.....	5

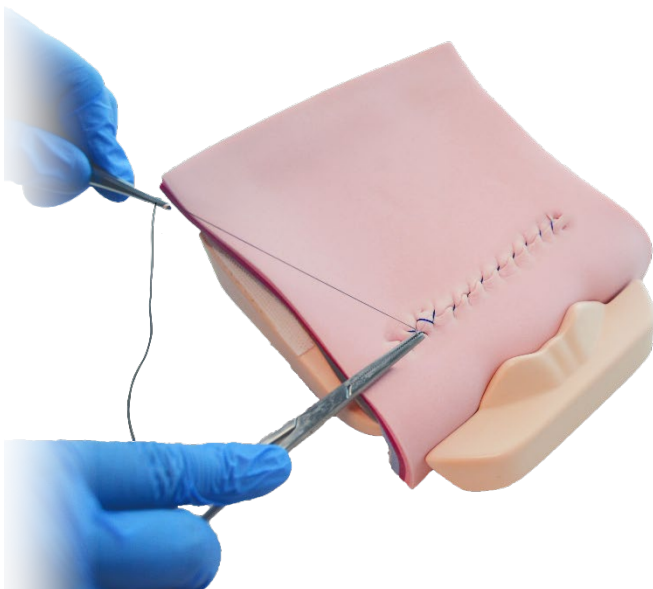KYOTO KAGAKU

## General Information

# Product Features and Precautions

### Before you start

MW74 is a trainer for hands-on training in incision and suturing of uterine wall during cesarean section procedures.

### Manufacturer's note

MW74 is designed for medical education training and research. Please do not use the product for other purposes. Any other use not in accordance with this manual is strongly discouraged. The manufacturer holds no responsibility for any accidents or damages resulting from such use.

### Features

- Multiple layers of uterine wall
- Realistic sensation of incision and suture
- Fetal head and maternal pubic bone are simulated

## Precautions


<ul style="list-style-type: none"><li>● <b>Handle with Care</b> This product is made from a special composition of resin. Please handle with care, ensure not drop it, and avoid shock as to not damage it</li></ul>	<ul style="list-style-type: none"><li>● <b>Cleaning and Care</b> Wipe the product with water or mild detergent. <b>Never wipe the product with thinner or organic solvent.</b></li></ul>
<ul style="list-style-type: none"><li>● <b>Do not leave printed materials on the product</b> Do not leave printed materials in contact with the surface of the product. Ink may be absorbed and stains will be irremovable.</li></ul>	<ul style="list-style-type: none"><li>● <b>Do not write on the product</b> Any ink on the product will be absorbed and become irremovable.</li></ul>
<ul style="list-style-type: none"><li>● <b>Avoid high temperature and humidity</b> Keep the product at room temperature, away from heat, moisture and direct sunlight.</li></ul>	<ul style="list-style-type: none"><li>● <b>Color change may occur</b> The color of the product may change over time. This does however not impact the performance.</li></ul>
<ul style="list-style-type: none"><li>● <b>Use the included storage bag for the pads</b> Be sure to use the included storage bag of nonwoven cloth to store the uterine wall pads. Don't use general plastic bags as oily exudate from the pad may damage the bag.</li></ul>	<ul style="list-style-type: none"><li>● <b>Store the suturing pad separately</b> Remove the uterine wall pad from the holder before storage. Leaving the pad on the holder for long time may lead to deformation of the pad. Store the pads separately in the included storage bag.</li></ul>

## Specifications

## Set Includes

### Set Includes



Before your first use, ensure all components listed below are included in the unit.

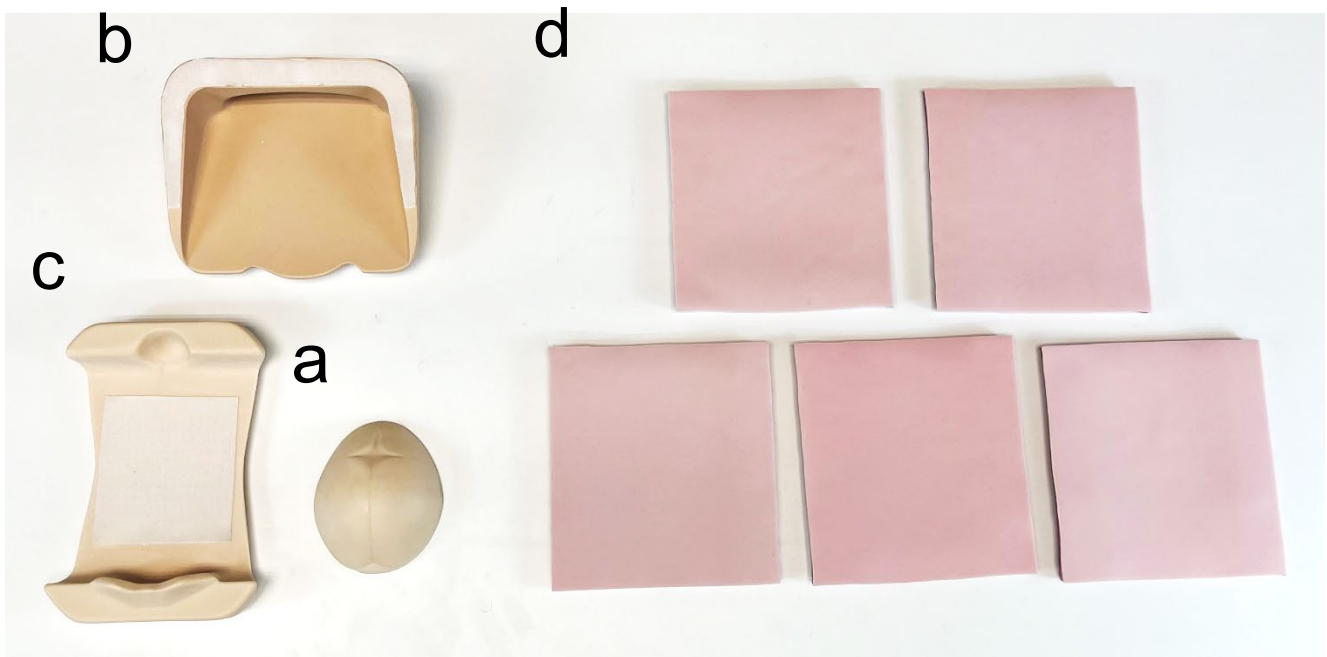a. fetal head	1
b. holder A	1
c. holder B	1

d. uterine wall pad	5
*consumable parts	

### Parts Names
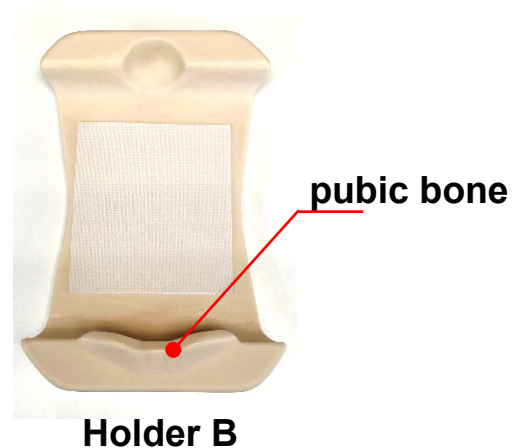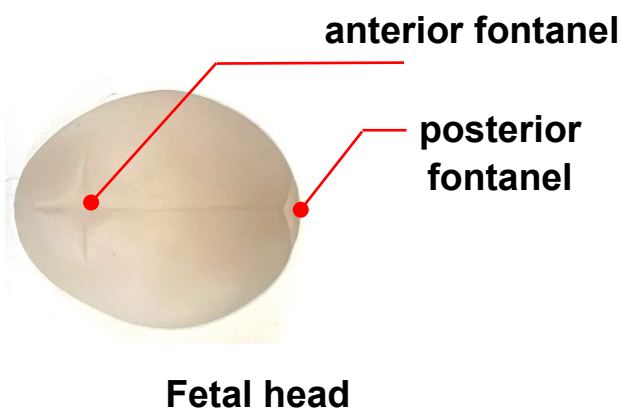
## Preparation

1. Disengage the holder A and holder B, which are fixed with Velcro tape.


2. Place the fetal head on the floor of the holder B.


2. Fix the uterine wall pad to the holder A with the Velcro tapes on the holder and on the back side of the pad.

\*make the lower end of the pad to roll up the lower edge of the holder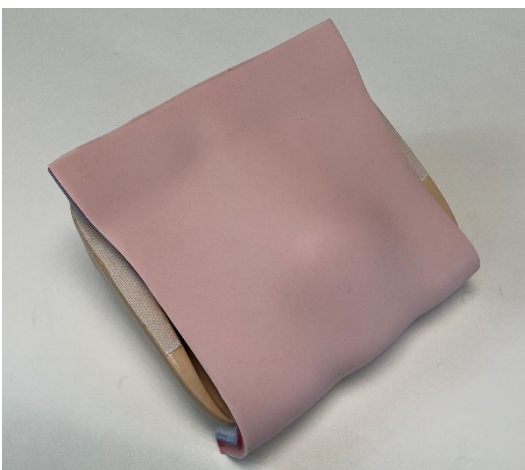

### Preparation

4. Fix the holder A with the pad on the holder B so that the pubic bone hold the uterine wall pad.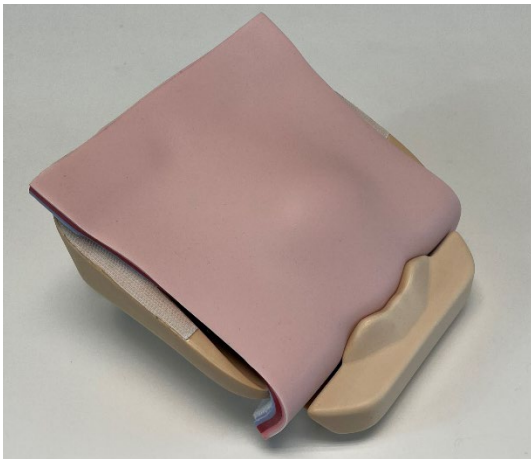

### Disassemble and Storage


1. Disassemble the unit following the steps in reverse of assembly process outlined on page 3-4.
2. Store the uterine wall pad separately in the included nonwoven bag.

## Consumables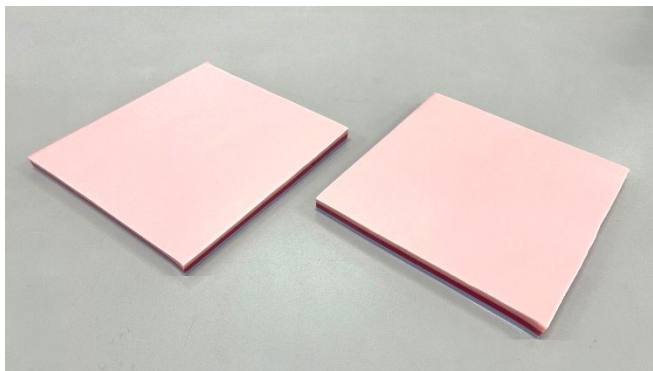

11479-010

Uterine wall pad for MW74 (set of 2)

### ■ USA, Canada, and South America

**KKAmerica**  
Kyoto Kagaku America Inc.

**WEB** • [www.kkamerica-inc.com](http://www.kkamerica-inc.com)

**E-MAIL** • [info@kkamerica-inc.com](mailto:info@kkamerica-inc.com)

TEL : 1-310-325-8860 FAX : 1-310-325-8867

3109 Lomita Boulevard, Torrance, CA 90505-5108, USA

### ■ Europe, Russia & Africa

**Kyoto Kagaku Europe GmbH.**

TEL : +49-69-5060-28160

De-Saint-Exupery-Str.10,60549 Frankfurt, Germany

### ■ Worldwide Inquiries and Orders

**KYOTO KAGAKU co.,LTD**

URL: <http://www.kyotokagaku.com> e-mail: [rw-kyoto@kyotokagaku.co.jp](mailto:rw-kyoto@kyotokagaku.co.jp)

TEL : +81-75-605-2510 FAX : +81-75-605-2519

15 Kitane-koya-cho, Fushimi-ku, Kyoto, 612-8388, JAPAN

The contents of the instruction manual are subject to change without prior notice.

No part of this instruction manual may be reproduced or transmitted in any form without permission from the manufacturer. Please contact manufacturer for extra copies of this manual which may contain important updates and revisions.

Please contact manufacturer with any discrepancies, typos, or mistakes in this manual or product feedback. Your cooperation is greatly appreciated.

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