


11489-000



MW85

Clinical Assessment Kit

-Surgical Wound-

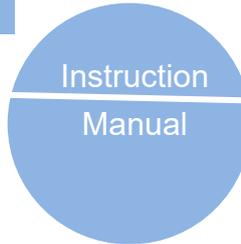

• **Don't wipe the kit using organic solvent**
Organic solvent damage the surface coloring and coating. For clearing, use alcohol or water.

• **Be sure to remove the kit from the manikin after use and store it in its original storage tray and bag.**
Leaving the kit adhered on a simulator or a manikin for or prolonged period of time may cause stain or deterioration of the surface.



Precautions

• **Do not use this product for purposes other than its intended use**
The product is designed for training of healthcare professionals. Usage for any other purpose or the way of use not described in this manual is strongly discouraged.

• **This product is to be used with Kyoto Kagaku Simulators and Mankins.**
The manufacturer cannot be held responsible for use with third-party products.

• **Don't scrub the surface**
Frication may cause deterioration of the surface or damage the coloring.

• **Do not let ink from pens, printed materials, this manual or other sources contact with the product's surface , as they cannot be cleaned off.**


• **Store dark, cool and dry space will help prevent the product's colors from fading and product from deforming.**

• **Never use the organic solvent such as paint thinner to clean the skin, as this will damage the simulator.**

• **The Kit best perform on a smooth and high-adhesion surface.**
Adhesion might possibly be insufficient on a textured or uneven surface.

■ Related product
11489-100 MW85 Clinical Assessment Kit -Cyanosis-

Set Includes

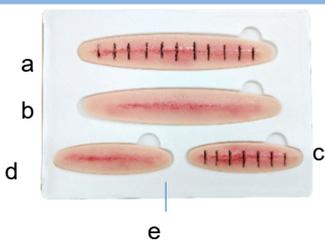

- a. wound pad large (with suture) 1
- b. wound pad large (without suture) 1
- c. wound pad small (with suture) 1
- d. wound pad small (without suture)..... 1
- e. storage tray 1
- f. storage bag..... 1



www.kyotokagaku.com rw-kyoto@kyotokagaku.co.jp

Kyotokagaku Head Office and Factories:

15 Kitane-koya-cho, Fushimi-ku, Kyoto, 612-8388, JAPAN, Tel: +81-75-605-2510, Fax: +81-75-605-2519

Kyoto Kagaku America Inc: All America regions

3109 Lomita Boulevard, Torrance, CA 90505-5108, USA,
info@kkamerica-inc.com Tel: 1-310-325-8860, Fax: 1-310-325-8867 (Toll free in North America: 877-648-8195)

Kyoto Kagaku Europe GmbH. Europe, Russia & Africa

De-Saint-Exupery-Str.10, 60549 Frankfurt, Germany, rw-kyoto@kyotokagaku.co.jp Tel: +49-69-5060-28160

Training and Maintenance

1. Clean the attachment surface.

Oil, grease, or dust on the target surface may inhibit proper adhesion.
Wipe the target surface with alcohol and allow it to dry before application.



2. Attach the wound pad to the simulator

The pads adhere directly to the simulator without the need for additional adhesive or tools, utilizing the silicone's inherent adhesive properties.

Gently press the pad onto the target area to secure it in place.



3. Conduct training

Once the pad is securely attached, move onto training session, including surgical wound cleansing and irrigation.



4. After training, return the pads to the storage tray and store it together in the storage bag.

Leaving the kit adhered on a simulator or a manikin for or prolonged period of time may cause stain or deterioration of the surface.



◆ To Restore Adhesion

When the adhesion has become weaker, clean the rear surface of the pad with water or gently wipe it with alcohol using a nonwoven, lint-free soft cloth.

To reduce stickiness on the wound-side surface, lightly apply talcum powder, taking care to avoid the adhesive surface.



***Never use paint thinner or other organic solvent, as it may remove the coloring on the simulated wound.**



◆ For Greater Reality



2 simulated pus
(100ml each)
11446-020



2 simulated exudate
(100ml each)
11446-030



2 simulated blood
(100ml each)
11446-040



Apply these liquid to simulate wound with bleeding and exudate.

For details contact your distributor or Kyoto Kagaku Co.,Ltd.

The contents of the instruction manual are subject to change without prior notice

No part of this instruction manual may be reproduced or transmitted in any form without permission from the manufacturer
Please contact manufacturer for extra copies of this manual which may contain important updates and revisions.

Please contact manufacturer with any discrepancies, typos, or mistakes in this manual or product feedback. Your cooperation is greatly appreciated.