

MW87

MW88



Caution

Do not mark on the model and other components with pen or leave any printed materials contacted on surface. Ink marks on the model cannot be removed.

Newborn II Ken • Rika



Instruction manual

Contents

- Before you start ----P.1
- Training -----P.2

Manufacturer's Note

Thank you for purchasing the "MW87 or MW88

This model is a versatile neonatal simulator designed to closely replicate the feel of a real newborn. It can be used for a wide range of training purposes, including bathing practice, dressing and undressing, umbilical cord care, and diaper changing.

Precautions

Do not use this product for any purpose other than its intended use. The manufacturer shall not be held liable for any damage or accidents resulting from use that does not follow the instructions provided in the user manual.


Features

- The neck is not fixed, allowing users to experience the natural head support required for a newborn.
- The ears are soft and suitable for bathing practice.
- The anterior and posterior fontanelles are accurately reproduced.
- The limbs can be flexed and extended freely.
- The umbilical cord is detachable.

Introduction

Set Includes

Please ensure that you have all components listed all.

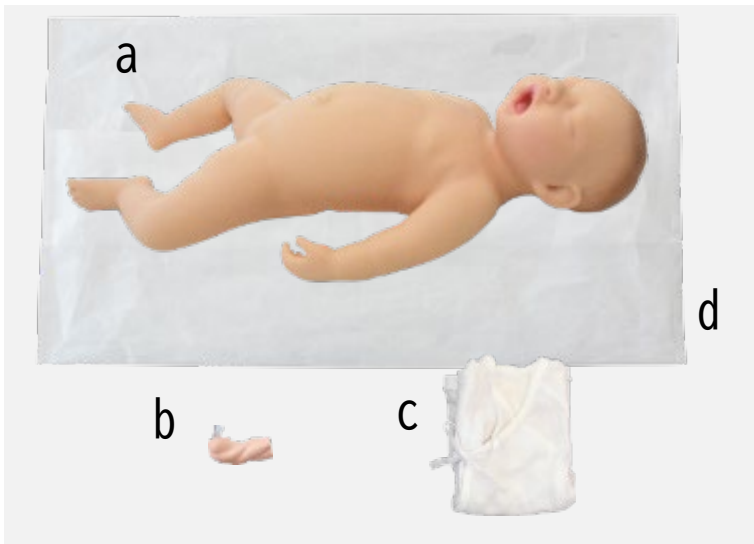

- a. Newborn model ----1
- b. Umbilical cord ----1
- c. Clothes ----1
- d. Model cover ----1

Precaution

- | | |
|--|---|
| <ul style="list-style-type: none"> ● Handle with care
Please handle the model with the same care as you would a real newborn.
As it is made of soft materials, do not pull strongly on the fingers or other parts. | <ul style="list-style-type: none"> ● Wipe off dirt using a neutral detergent or wet wipes.
Clean the model with water or a neutral detergent.
Do not use alcohol or organic solvents such as thinner under any circumstances. |
| <ul style="list-style-type: none"> ● Do not place printed materials on the surface of the model.
Ensure that printed materials do not come into direct contact with the model surface, as the ink may be absorbed into the resin and become impossible to remove. | <ul style="list-style-type: none"> ● Do not write on the model with ballpoint pens or markers.
Writing on the model, dressings, or tape with markers or ballpoint pens may cause the ink to be absorbed and become impossible to remove.
Also, avoid using substances such as povidone-iodine, as they may cause staining or discoloration. |
| <ul style="list-style-type: none"> ● Store in a cool, dry place away from high temperature and humidity.
Applying a thin layer of commercially available baby powder to the model surface helps prevent dirt and keeps it clean. When storing, avoid locations with high temperature, high humidity, or direct sunlight, as these may cause deformation or damage. | <ul style="list-style-type: none"> ● The surface may become discolored.
Discoloration may occur due to long periods of non-use or natural aging; however, this does not affect functionality. |
| <ul style="list-style-type: none"> ● After practice, remove the umbilical cord from the model before storing it.
When taking the model out of the cover, the umbilical cord may catch and could damage the model. | |

Training session

Training Skills

- Bathing
- Exercise
- Diaper changing, clothing
- Umbilical cord care

Bathing

You can practice preventing water from entering the ears by gently pressing in front of and behind the ears with your fingers. Handle the model gently, as you would a real newborn, taking care to support the neck appropriately. After practice, thoroughly wipe off any water and apply powder.



Exercise

The limbs can be flexed and extended. During exercise practice, handle the model gently, as you would a real newborn.



Diaper changing/clothing

You can practice diaper changing and dressing/undressing.



Umbilical cord care

The umbilical cord is removable.





WEB:www.kyotokagaku.com e-mail:rw-kyoto@kyotokagaku.co.jp

Worldwide Inquiries & Ordering

Kyotokagaku Head Office and Factories:
15 Kitanechoya-cho, Fushimi-ku, Kyoto, 612-8388, JAPAN
Tel: +81-75-605-2510 Fax: +81-75-605-2519

All America regions:USA, Canada, and South America



WEB:www.kkamerica-inc.com E-MAIL:info@kkamerica-inc.com
3109 Lomita Boulevard, Torrance, CA 90505-5108, USA
Tel: 1-310-325-8860 Fax: 1-310-325-8867
(Toll free in North America:877-648-8195)

Europe, Russia & Africa

Kyoto Kagaku Europe GmbH.

TEL:+49-69-5060-28160
De-Saint-Exupery-Str.10, 60549 Frankfurt, Germany

The contents of the instruction manual are subject to change without prior notice.

No part of this instruction manual may be reproduced or transmitted in any form without permission from the manufacturer.

Please contact manufacturer for extra copies of this manual which may contain important updates and revisions.

Please contact manufacturer with any discrepancies, typos, or mistakes in this manual or product feedback. Your cooperation is greatly appreciated.

2026/03