


MW5 Tube Feeding Routes Model

Instruction Manual

This model (MW5) is median sagittal plane model to help trainees anatomical understanding.

Training items

- ① Anatomical understanding
- ② Confirmation of the catheter tube position: route and distance
- ③ Intranasal and oral feeding tube insertion
- ④ Tracheotomy tube setting
- ⑤ PEG tube setting

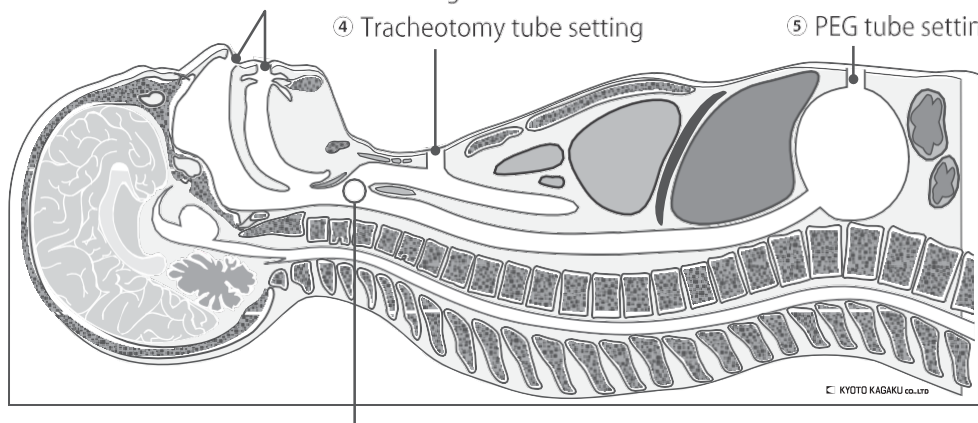

There is the hole in order to change the routes of the feeding tube. It can show also the false insertion to the trachea as well as correct one.

This model can be used at both lateral position and sitting position by replacing the stand.
The distance from nostrill and oral to the stomach is 45~50cm as well as real patient.

【The recommended size of the tube and the catheter】

Intranasal and oral feeding tube: 14Fr.
Tracheostomy tube: 8.00ID (11.0mm OD)
PEG tube: 20Fr. X 3.4 cm

【Dimensions】

approx. 63×23×2.5cm, 1.1kg

Caution

- **Don't administrate real liquid foods (formulas).**
This model is made for the sake of the confirmation tube feeding routes and anatomical understanding.
- **Don't leave the printed materials contacted on this model surface.**
Ink marks on the models will be irremovable.
- **Don't mark on the model with pen or other writing materials.**
Ink marks on the models will be irremovable.
- **This model may be cleaned with mildly soapy water or diluted detergent.**
The model may be cleaned with a wet cloth, if necessary, using mildly soapy water or diluted detergent.
Never use ethanol or organic solvent like paint thinner.
- **Storage in a dark, cool space.**
Storage in a dark, cool space will help prevent from being transformed or having troubles.
- **There is the case that the surface changes color.**
Changing color by secular variations will occur when it is not used for a long term though it does not have any problem for training.

• For inquiries and service, please contact your distributor or KYOTO KAGAKU CO., LTD.

2026.08.07

 **KYOTO KAGAKU CO., LTD**



<http://www.kyotokagaku.com>



rw-kyoto@kyotokagaku.co.jp

Contact Address:

Main Office and Factory

15 Kitanehoya-cho Fushimi-ku Kyoto 612-8388, Japan
Phone : 81-75-605-2510
Fax : 81-75-605-2519

LA Office (for USA, CANADA and Mexico customers)

Kyoto Kagaku USA 3109 Lomita Blvd, Torrance, CA 90505
Phone: 310-325-8860 Fax: 310-325-8867