

Storage of the Chest Plates for PH-1

The surface of the chest plates is covered with clear coating to prevent scratches and stains. Although the coating is durable for general handling, at times cracks or wrinkles might possibly occur due to bending movement of the chest plate. We hope you would stay assured that these cracks are only the appearance on the surface and do not affect the radiographs or CT images, though, follow the instructions below to keep the plates at its best condition longer.

- Hold two **central areas** of the plate (at the neck and the waist) when handling it.

Do not hold its sides.

- When putting the plates temporarily, keep them standing upright. Do not lay down the plates horizontally without the **supporting base**.

Do not press on the plates or put heavy items on them.

- For storage, be sure to place the plates on the supporting bases.

- Store in a place away from high temperature, humidity and direct sunlight.

- Do not drop the plates.

- Do not drop any sharp or hard materials on the chest plates.

