



Caution

Do not mark on the model and other components with pen or leave any printed materials contacted on surface. Ink marks on the model cannot be removed.

PH-72

MRI Breast QA phantom

Instruction manual

Contents

- Set Includes-----P.1
- Introduction-----P.2
- Preparation-----P.4




Read before use

The product is intended for use of medical professionals only. Any other use, or any use not in accordance with the enclosed instructions, is strongly discouraged. The manufacturer cannot be held responsible for any accident or damage resulting from such use. Please use this product carefully and refrain from subjecting to any unnecessary stress or wear. Should you have any questions on this simulator, please feel free to contact our distributor in your area or KYOTO KAGAKU at any time. (Our contact address is on the back cover of this manual)

Set includes

Set includes

Before your first use, please ensure that you have all components listed below.

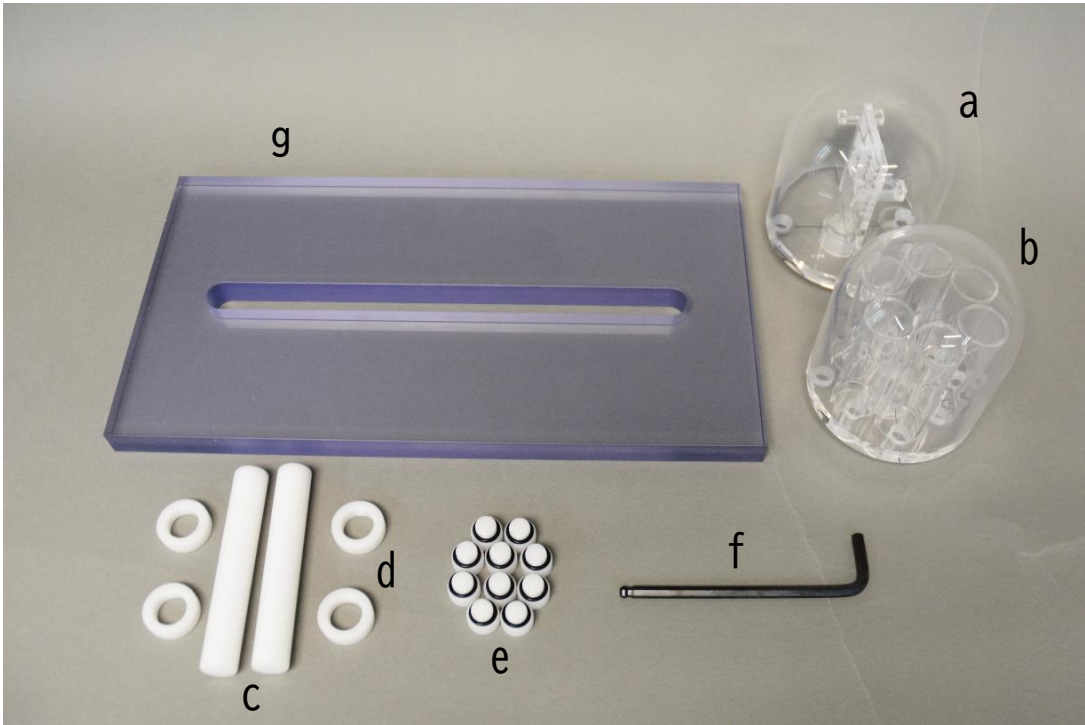

- | | |
|----------------------------|---|
| a. Slit phantom -----1 | e. Screw stopper (with hexagonal pit) -- 10 |
| b. Contrast phantom ---- 1 | f. Hexagonal wrench ---- 1 |
| c. Adjustment bolt ---- 2 | g. Supporting plate ---- 1 |
| d. Support ring ---- 4 | |

DOs and DON'Ts


<ul style="list-style-type: none"> ● Handle the phantom and the components with care Be sure not to drop or add shock to the manikin and its components, since the product uses special hard resins and heavy, such shock may cause damage to the products or environment. 	<ul style="list-style-type: none"> ● The product may be cleaned with a wet cloth if necessary, using mildly soapy water or diluted detergent. Never use the organic solvent such as paint thinner to clean the skin, as this will damage the simulator.
<ul style="list-style-type: none"> ● Storage dark, cool and dry space will help prevent the product's colors from fading and product from deforming. 	<ul style="list-style-type: none"> ● Do not let ink from pens, oriented materials, this manual or other sources contact with the product's surface, as they cannot be cleaned off.

Introduction

Specifications: Slit Phantom



Drawing

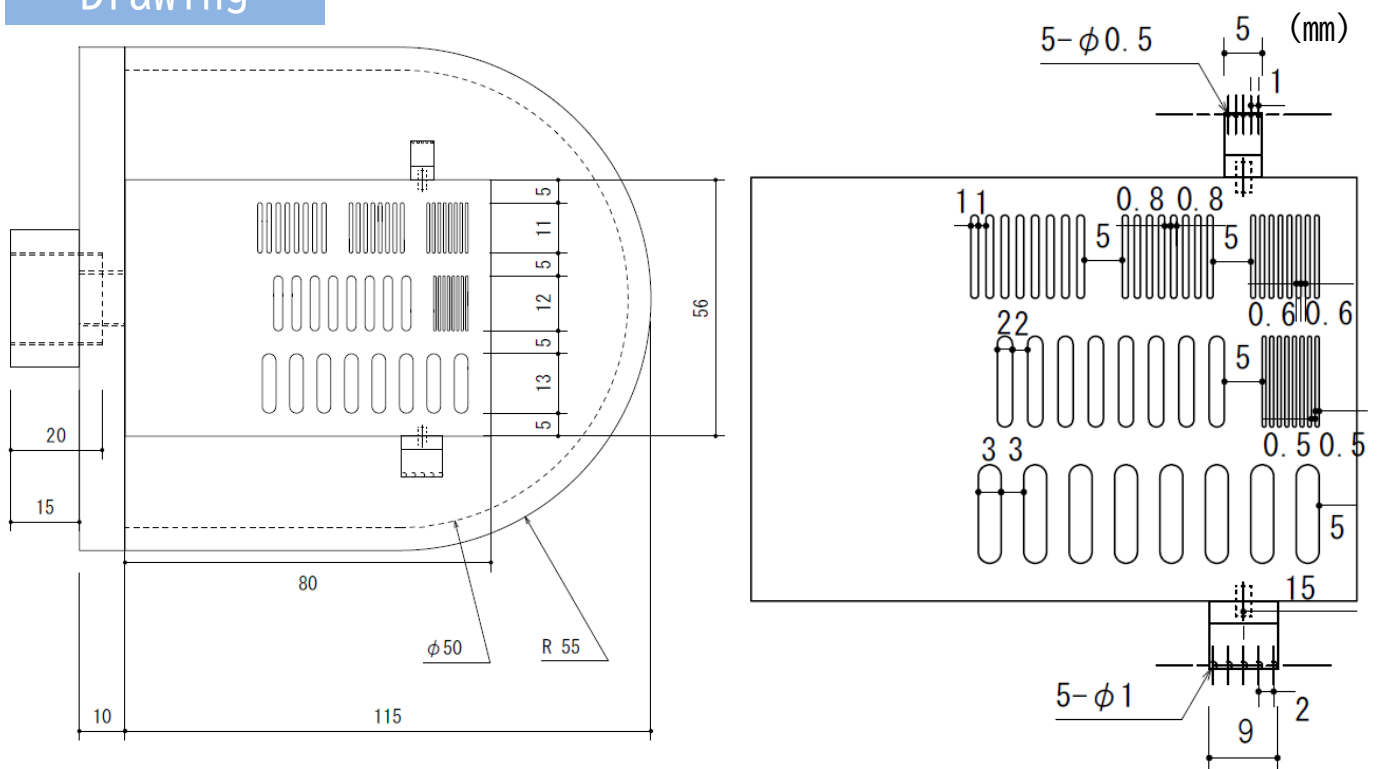


Preparation

Assembling the Phantom

1. Fill the phantom with RI solution and seal the screw stopper.





2. Attach the adjustment bolt to the phantom.

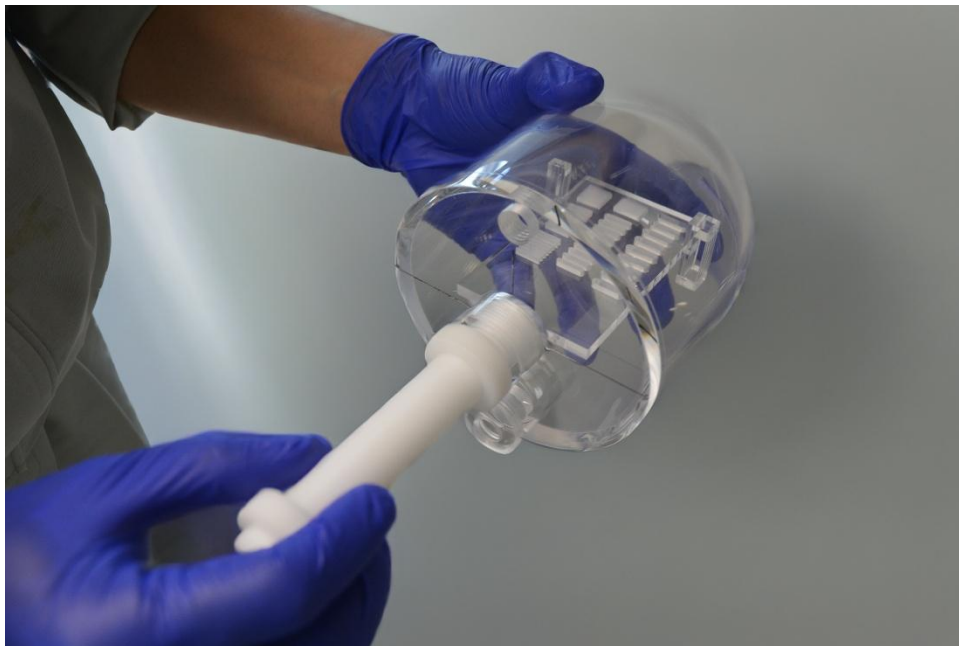

Preparation

Assembling the Phantom

3. Attach the phantoms to the supporting plate.
Fix the phantoms by clamping the supporting plate with each two support rings.

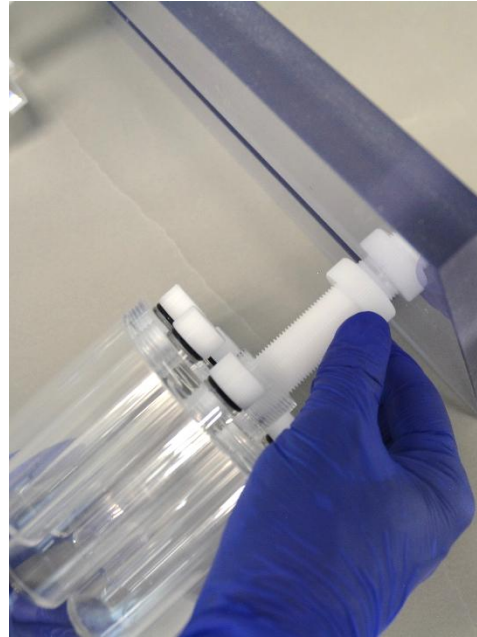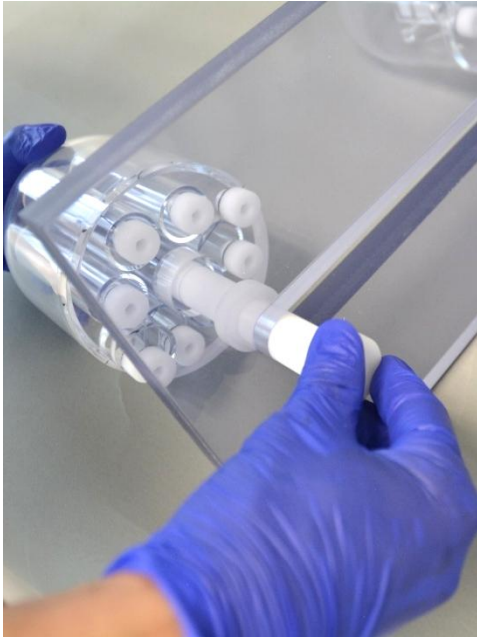

For inquiries and service, please contact your distributor or KYOTO KAGAKU CO., LTD.



WEB:www.kyotokagaku.com e-mail:rw-kyoto@kyotokagaku.co.jp

Worldwide Inquiries & Ordering

Kyotokagaku Head Office and Factories:
15 Kitanechoya-cho, Fushimi-ku, Kyoto, 612-8388, JAPAN
Tel: +81-75-605-2510 Fax: +81-75-605-2519

All America regions:USA, Canada, and South America



WEB:www.kkamerica-inc.com E-MAIL:info@kkamerica-inc.com
3109 Lomita Boulevard, Torrance, CA 90505-5108, USA
Tel: 1-310-325-8860 Fax: 1-310-325-8867
(Toll free in North America:877-648-8195)

**Europe, Russia & Africa
Kyoto Kagaku Europe GmbH.**

TEL:+49-69-5060-28160
De-Saint-Exupery-Str.10, 60549 Frankfurt, Germany

2025/11

The contents of the instruction manual are subject to change without prior notice.
No part of this instruction manual may be reproduced or transmitted in any form without permission from the manufacturer.
Please contact manufacturer for extra copies of this manual which may contain important updates and revisions.
Please contact manufacturer with any discrepancies, typos, or mistakes in this manual or product feedback. Your cooperation is greatly appreciated.