

PH-1 Chest Phantom N1 LUNGMAN <Capillary vessels repair kit>

Capillary vessels of the N1 Lungman may come off by handling.
To repair the vessels, follow the instruction below.

Kit includes: Adhesive (Krazy Glue), Petri dish, Tweezers, Waterproof sheet, Skewer

What you need to prepare: Accelerator for the adhesive (recommended: Aron Alpha Setter)

<Please note>


This repair kit and instruction are to be used for repair of the phantom vessels only.

Do not use this kit to attach the simulated tumors.

If you attach the simulated tumors with the included adhesive, they will become irremovable.

Also, read the instruction manual of the adhesive and the accelerator.

How to repair

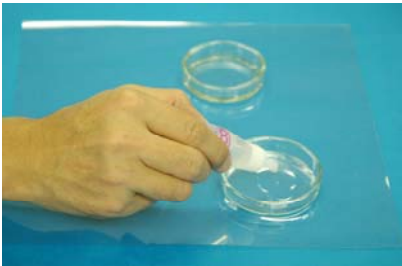

Spread the waterproof sheet on the table and put two Petri dishes on it. Put a little amount of adhesive and accelerator into each separate dish.



Cut off the end of the capillary vessel to remove the remaining old adhesive.



Using the skewer, a) put a small amount of adhesive to the end of capillary vessel and b) put a small amount of accelerator where you are going to attach it. Please aware that excessive amount of adhesive could disturb the process of adhesion.



Hold the capillary vessel by the tweezers and attach it to where you put the accelerator. Hold it there for 4–5 seconds until the vessel is firmly fixed.