

Instruction Manual

Please read this manual carefully before use.!

Thyroid Biopsy Phantom

Pre-use Checks and Precautions

| | | |
|--------------|-----------|----------------------------------------|
| Product Code | 41957-020 | Transparentbiopsy block for US-25 1 pc |
| | 41957-030 | opaque biopsy block for US-25 1 pc |


Manufacture's Note

This product is designed to provide training in ultrasound guided thyroid biopsy.

Any use not in accordance with the enclosed instructions is strongly discouraged.



| | |
|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <p>●Handle with care The materials of the products are a special composition of soft resin. Handle the parts with utmost care at all times</p> | <p>●Remove the ultrasound gel after use After use, promptly remove the ultrasound gel using a moistened tissue or soft cloth. NEVER use organic solvents to clean the components.</p> |
| <p>●Ink marks on the soft surface won't be removable Don't mark on the product and components with pen or leave any printed materials in contact with their surface.</p> | <p>●Surface color might change overtime Discoloration may occur over time or during prolonged periods of non-use; however, this does not affect functionality.</p> |
| <p>●Remove the protective sheet before use. The surface of the pads is covered with transparent protective sheets. After cleaning, reattach the transparent protective sheet to the surface of the pad.</p> | <p>●Avoid heat and humidity for storage Store the components in the storage case at room temperature, away from heat, moisture, and direct sunlight, to prevent deformation or material deterioration..</p> |
| | <p>●Recommended size of the needle: 23G Using biopsy needles of not recommended size may result in:</p> <ul style="list-style-type: none"> •difficulty in aspirating the liquid •damage of the block |

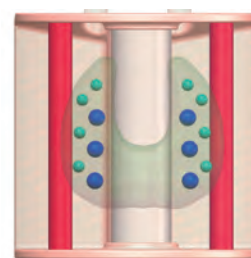

【Training】

1. Apply ultrasound gel on the block.
2. Place the transducer and visualize the targets.
3. Puncture the target and aspire the liquid in the target.

【After Training】

1. Gently remove any residual gel from the surface with a moistened tissue or soft cloth.

Targets Blue Approx. \varnothing 7 mm 6 pieces
*Liquid Green Approx. \varnothing 5 mm 8 pieces

Vessels Red Approx. \varnothing 10 mm 2 pieces
(left and right) *Solid resin (not liquid)

For inquiries and service, please contact your distributor or KYOTO KAGAKU CO., LTD.



KYOTO KAGAKU co.,LTD



www.kyotokagaku.com



rw-kyoto@kyotokagaku.co.jp

Kyotokagaku Head Office and Factories: 15 KitaneKoya-cho, Fushimi-ku, Kyoto, 612-8388, JAPAN
Tel: +81-75-605-2510 Fax: +81-75-605-2519

All America regions: USA, Canada, and South America

Kyoto Kagaku America Inc.: 3109 Lomita Boulevard, Torrance, CA 90505-5108, USA
Tel: 1-310-325-8860 Fax: 1-310-325-8867 E-MAIL: info@kkamerica-inc.com
(Toll free in North America: 877-648-8195)

Europe, Russia & Africa

Kyoto Kagaku Europe GmbH: De-Saint-Exupery-Str.10, 60549 Frankfurt, Germany
TEL: +49-69-5060-28160