


**Caution:**

Don't mark on the phantom with pen or leave printed materials contacted on its surface.  
Ink marks on the phantom will be irremovable.

US-8

# Pediatric FAST/Acute Abdomen Phantom

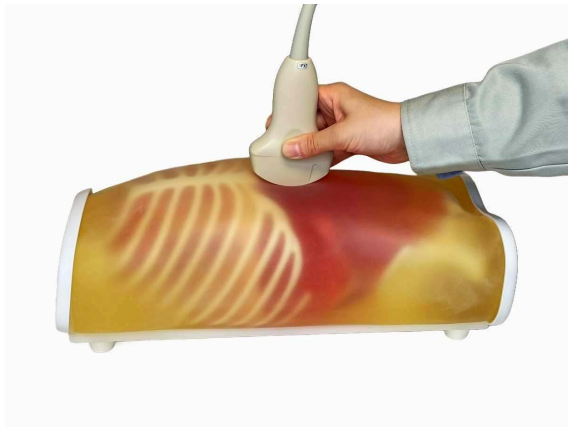

## Contents

- Please read before training
  - General information . . . . . P.2
- Preparation
  - 1、 Befor training . . . . . P.2
- Training
  - 2、 Training session . . . . . P.2
  - 3、 After training . . . . . P.2


This product is intended to be used for medical training. Do not use for other than the intended purpose.

# Please read Preparation/ Training

# General information Training session

## Set includes:

Before your first use, please ensure that you have all components listed below.

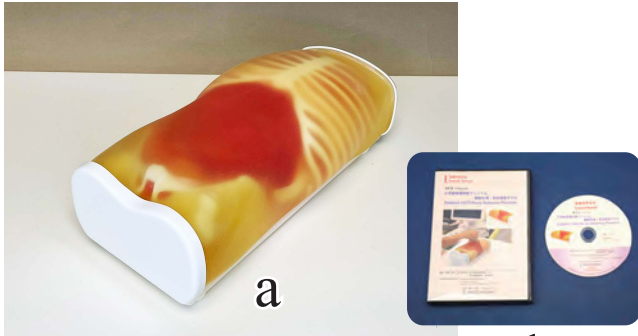

a. pediatric FAST/Acute abdomen phantom	1
storage case	1
b. DVD tutorial manual	1
c. talcum powder	1
Instruction manual (this leaflet)	1

## ⚠ DOs and DON'T s

### DO s

#### Handle with care

The materials for phantom and models are special composition of soft resin. Please handle with care at all times.

#### Cleaning and care

Clean the phantom completely every time after the training.

The remaining gel may deteriorate the phantom. Keep the phantom at room temperature, away from heat, moisture and direct sunlight.

### DON'Ts

Never wipe the phantom or models with thinner or organic solvent.


Don't mark on the phantom with pen or leave printed materials contacted on their surface.

Ink marks on the models will be irremovable.


Please note: The color of the phantom may change over time, though, please be assured that this is not deterioration of the material and the ultrasonic features of the phantom stay unaffected.

This phantom has a thin layer of coating for protective purposes which may cause minor wrinkling. However, it has no effect on the ultrasound image.

## 1 Before training

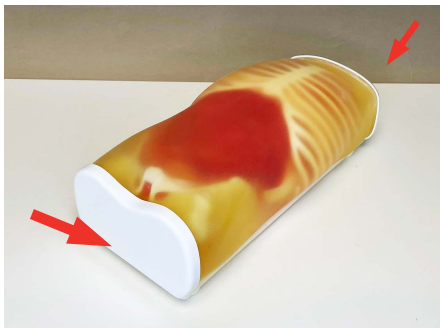

① Move the phantom, be sure to grip both handles at the base.

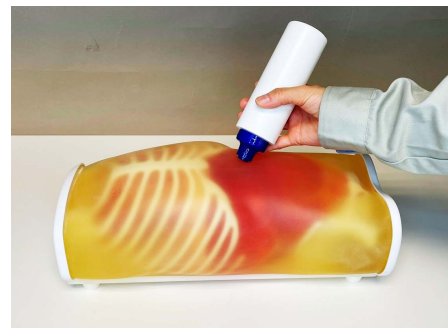

② Spread ultrasound gel over the phantom body.


## 2 Training session



The phantom can be scanned with actual ultrasound devices.

\* The map of simulated hemorrhages and tumors is on the backside of this manual.


## 3 After training

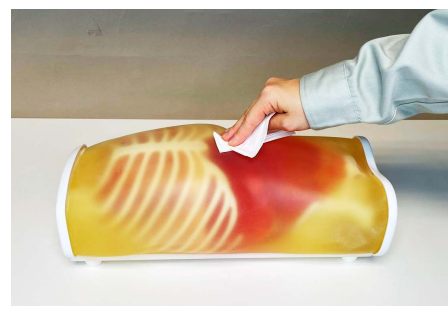

Wipe the remaining gel completely by wet cloth. If remaining gel gets dried up on the body, it may cause scratches and tears on the phantom material.

**⚠ Caution**

Don't mark on the phantom with pen or leave printed materials contacted on its surface. Ink marks on the phantom will be irremovable.

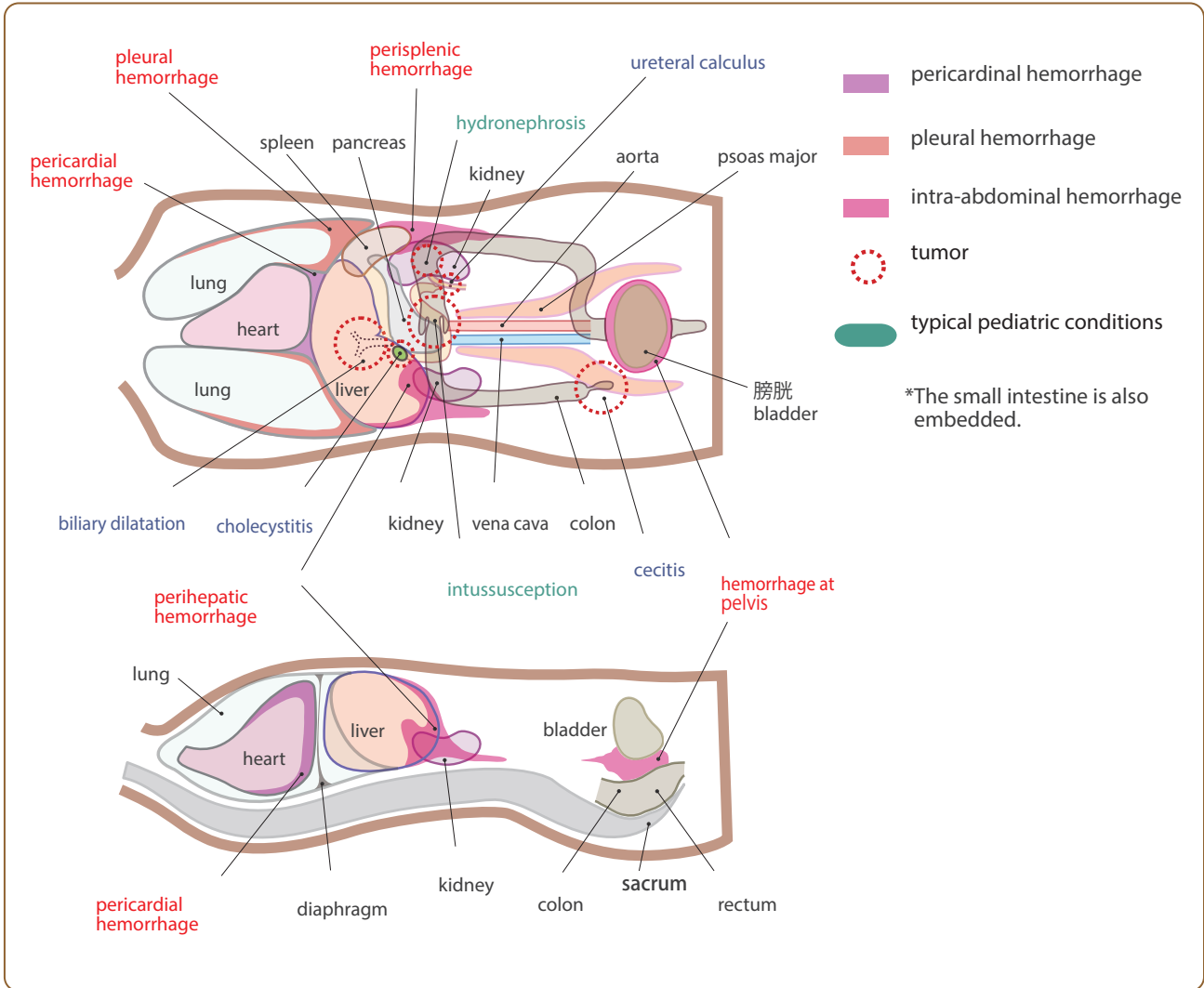

The contents of this manual are subject to change without prior notice.

No part of this instruction manual may be reproduced or transmitted in any form without permission from the manufacturer. Please contact manufacturer for extra copies of this manual which may contain important updates and revisions.

Please contact manufacturer with any discrepancies, typos, or mistakes in this manual or product feedback. Your cooperation is greatly appreciated.

**KYOTO KAGAKU co.,LTD**

URL : <http://www.kyotokagaku.com>  
e-mail : [rw-kyoto@kyotokagaku.co.jp](mailto:rw-kyoto@kyotokagaku.co.jp)

**■ Main Office and Factory**

15 Kitanechoya-cho Fushimi-ku Kyoto 612-8388, Japan  
Telephone : 81-75-605-2510  
Facsimile : 81-75-605-2519

**■ KyotoKagaku America Inc.**

USA,Canada,and South America  
3109 Lomita Boulevard, Torrance, CA 90505 ,USA  
Telephone : 1-310-325-8860  
Facsimile : 1-310-325-8867



15 Kitanechoya-cho Fushimi-ku  
Kyoto, Japan 612-8388  
Telephone : +81-75-605-2510  
F a x : +81-75-605-2519

Dear customer,

**Note**

Thank you for choosing our ultra sound phantom.

There may be a case that the surface of phantom is in a wet condition.

The ultra sound image will not be affected. In such case, wipe the surface with medical use alcohol. **Do not wipe the surface with strong solvent such as benzene.**

If you have any questions regarding this matter, please do not hesitate to contact us at any time.

Faithfully yours,

Kyoto Kagaku Co., Ltd.

Email: [rw-kyoto@kyotokagaku.co.jp](mailto:rw-kyoto@kyotokagaku.co.jp)