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Caution:

Don't mark on the phantom with pen or leave printed materials contacted on its surface.
Ink marks on the phantom will be irremovable.

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Contents

- Please read before training
General information P. 2
- Preparation
1 Befor training P. 2
- 2、 Training session P. 2

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This product is intended to be used for medical training. Do not use for other than the intended purpose.

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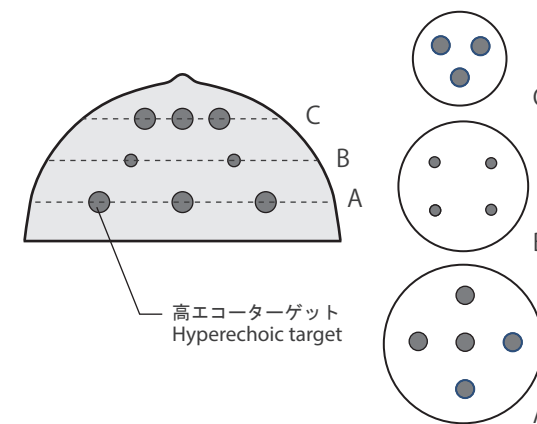
■Specifications

Size of targets: 6 mm dia. and 10mm dia.
Type of targets: Hyperechoic (blue)
Number of targets: 12 pieces
Phantom size: 136mm dia.x70mmH
Phantom weight: 700g

■模擬腫瘍配置図 Targets in the breast phantom

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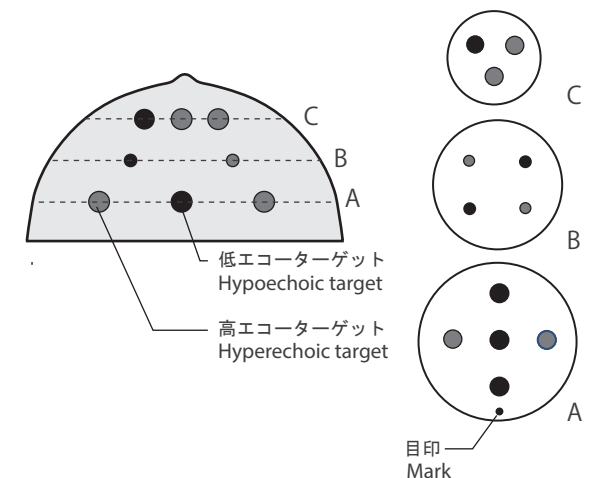
■Specifications

Size of targets: 6 mm dia. and 10mm dia.
Type of targets: Hyperechoic (blue),
Hypoechoic (red)
Number of targets: 12 pieces
Phantom size: 136mm dia.x70mmH
Phantom weight: 700g

■模擬腫瘍配置図 Targets in the breast phantom

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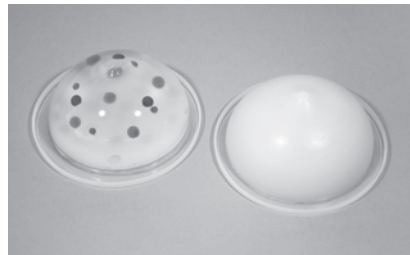
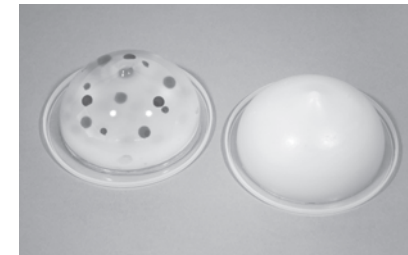
1

Please read Preparation/ Training

General information Training session

Set includes

- Duo Set: 1387-000
(transparent + opaque type)
- Transparent Set: 11387-100
(1 pair of transparent type)
- Opaque Set: 11387-200
(1 pair opaque type)

11387-000 (transparent + opaque type)

⚠ Caution

- This is a disposable product.
- The aqueous material used in this product contracts as time passes after the package is opened.
- After opening the package, store the product along with the case within a sealed bag or a similar container.
(This method is only for temporary storage. You can delay, but cannot stop contraction and deformation of the product by this storage method.)
- No fluid injection is allowed.
- Do not stretch or press hard on the puncture area.
- Dispose the product as general waste after use.

1、Befor training

① Remove the back cap on the case.

② Remove the film from the case.

③ Take out the product from the case while inserting a finger between the case and the product to let the air get into between them, then put the product on the back cap.

2、Training session

① Conduct puncture training under ultrasonic guide after applying water or ultrasonic gel on the product.

※Temporary storage

After you have removed the transparent film from the case, store the product along with the case after closing the back cap in a sealed bag or similar container. (The product can be stored only for a short while by this method.)

2